TVDirect[™] model TV-95 for private TV listening

Hear everything without disturbing others

Features

- It's wireless, it rests naturally and gently on your shoulders lets you listen in great comfort.
- Proprietary High Definition Sound (HDS) Technology amplifies (115 dB spl) and delivers CD quality stereo sounds and life-like sound effects direct to your ears..
- Hi-fidelity and passive noise isolation speakers stay gently in your ear without creating pressure or discomfort.
- It's so light weight that you feel like you're not wearing anything.
- Adjustable Digital Tone control enhances dialog clarity and intelligibility to suit your hearing.
- Long battery life—up to 24 hours of listening time per full-charge.
- Optional tele-coils and neckloop for people wearing t-coil hearing aid.

Model TV-95

SERENE

TVDirect

Model TV-95

Operating Guide

Thank you for purchasing this TV-Direct™ listener. Our proprietary High Definition Sound (HDS) technology converts any TV audio output into an infrared (IR) light beam for wireless transmission to a receiver. This unique technology dramatically broadens the acoustic sweet spot of your living room. This means you can clearly hear all dialogues and stereo sounds anywhere in the room.

Please read this operating guide carefully before use and keep it for future reference.If you have any questions, please feel free to contact our Customer Care Department.

Phone:(562) 407-5400 Fax:(562) 483-2087

Toll-free Help Line:866-376-9271 (9AM-5PM PST,M-F)

IMPORTANT SAFETY INSTRUCTIONS

1. Please read all these instructions carefully before using this TV listener.
2. Always keep the system dry and do not expose it to direct sunlight.
3. Furniture polish, over time, may disintegrate the rubber feet of the transmitter which may stain your furniture. To avoid a potential stain, you may want to place a transmitter on a pad or mat.
4. This listener allows you to amplify sounds to a loud volume, but listening to earphones at a high volume over an extended period of time may damage your hearing; especially in children. Thus, we suggest that you set the volume to a level that is comfortable to your hearing.
5. TVDirect™ uses rechargeable batteries for operation. please observe the safety and handling instructions for the rechargeable batteries. Always recycle batteries through your local waste recycler.
6. This listener is intended for home and personal use only.

BATTERY SAFETY AND DISPOSAL

CAUTION

TV-Direct™ is supplied with two rechargeable NiMH batteries. Do not replace with regular non-rechargeable batteries. Recharging nonrechargeable batteries may cause them to explode and release dangerous chemicals; causing serious injury to the user and others.

Please do not dispose of used batteries in the household trash. Please take the batteries to a retail or community collection point for recycling.

RECYCLING INSTRUCTIONS

Help protect the environment!


Please take the time to dispose any used equipment properly.

Product Recycling for Customers in the European Union:


Please do not dispose of your old equipment in the household trash. Please take the equipment to an nearby recycle center, or return the product to the factory for proper disposal.

PACKAGE CONTENT CHECK

Your package includes all items as shown here. If there is anything missing, please contact our Customer Care Department immediately.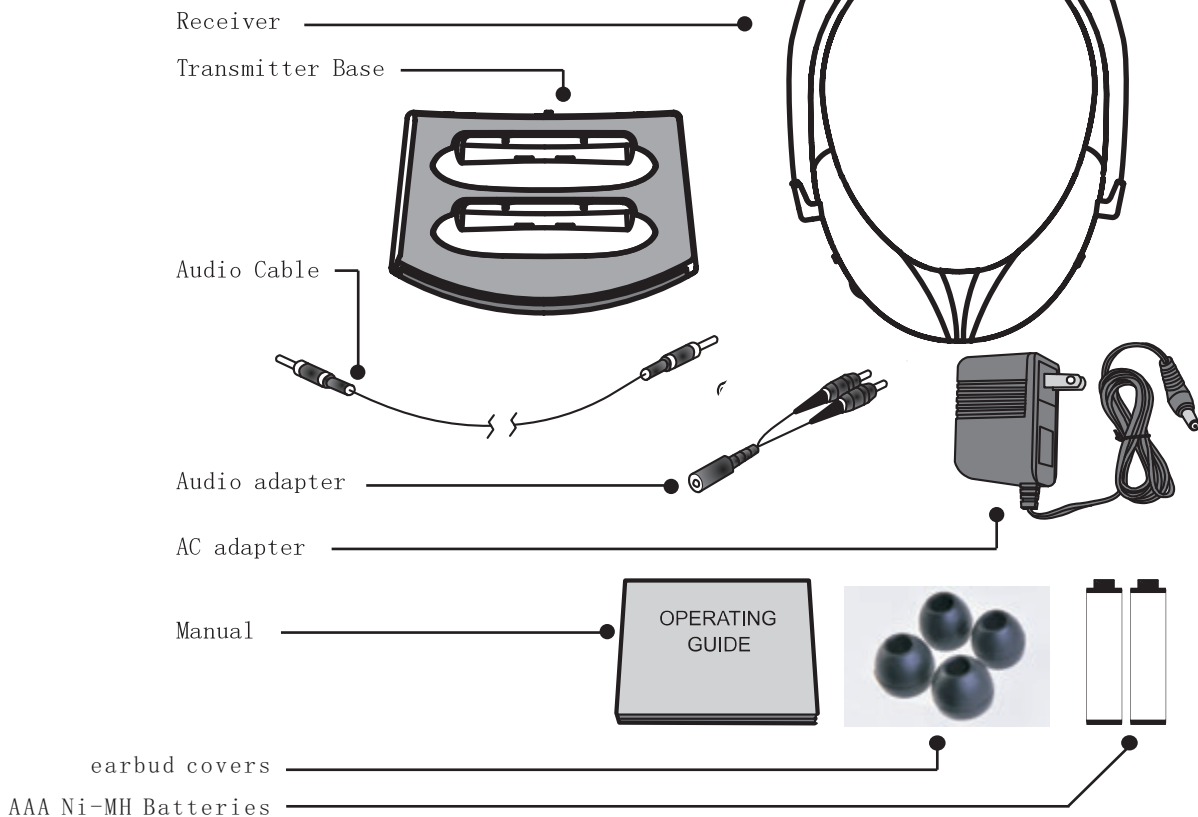


FEATURE IDENTIFICATION

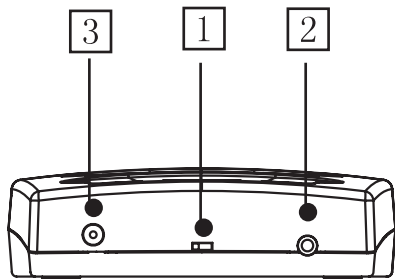
TRANSMITTER BASE

1. Stereo/Mono/Microphone Selector switch.
2. Audio-in jack for all audio inputs.
3. AC adapter input jack.
4. IR transmitter window.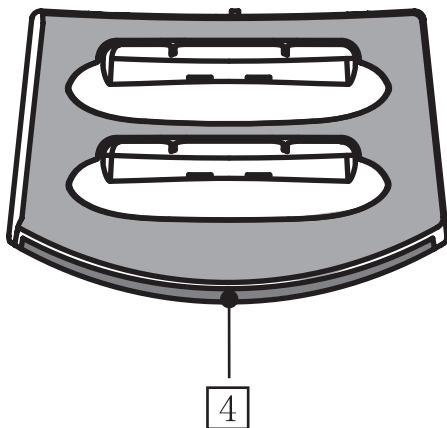


FEATURE IDENTIFICATION

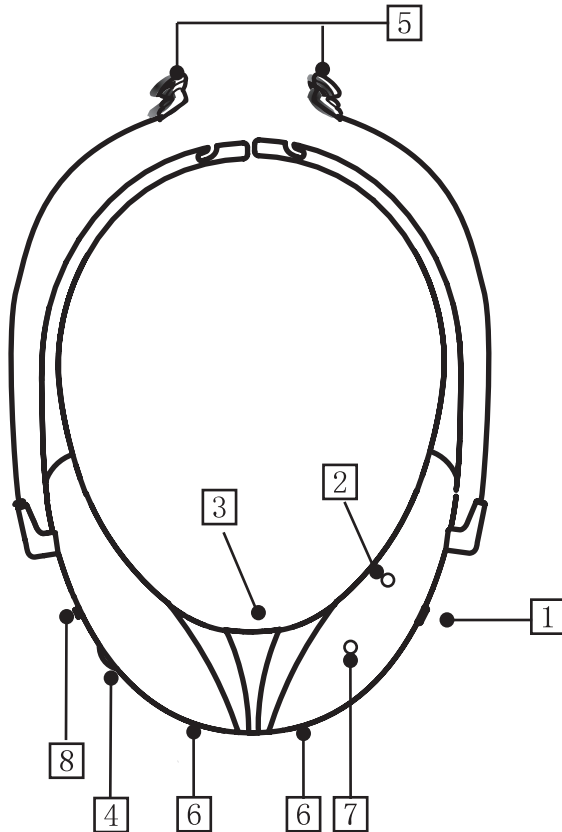RECEIVER

1. Receiver On/Off pushbutton.
2. Receiver On/Off power supply indicator.
3. Battery compartment (rear).
4. Volume control.
5. Left and right earphones
6. Battery charging contacts.
7. Charging indicator light.
8. Tone control to enhance dialog clarity and intelligibility (Low-Mid-High).

CONNECTING THE TRANSMITTER TO YOUR TV, CABLE BOX, OR OTHER SOUND SOURCE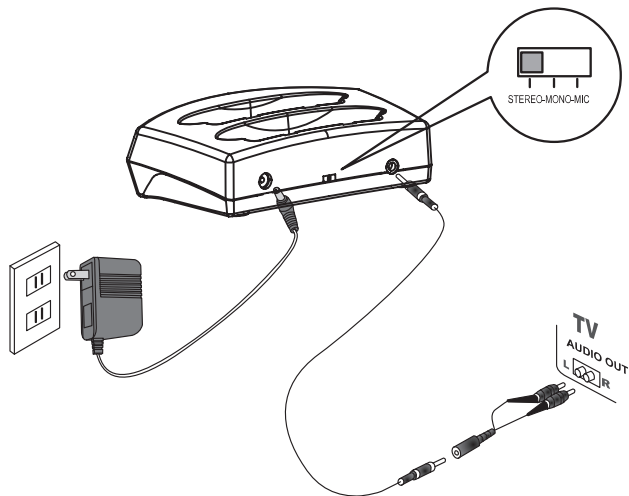


Option 1 (Recommended)

1. Set the selector switch on the transmitter base to stereo mode.
2. Use the provided audio cable (and audio adapter) to connect "TV Audio" on the transmitter base to the **audio-out** jacks of your TV (or cable box) as shown.
3. Connect and plug in the AC Adapter as shown. The IR diodes glow behind the IR transmitter window as long as the TV/sound source is **ON** to indicate that signals are being transmitted.
4. Set the TV volume to a comfortable level. On many TV sets, you may "Mute" the TV speaker output and still be able to hear via the TV-Direct.

Option 2 Same procedures as Option 1, except that you plug the audio cable into the headphone port (usually located at the front of your TV). If you use this connection mode, adjust the volume of the headphone output to a medium level. If this connection mutes the TV speaker sound, you can deactivate the muting function by using the TV setting menu.


Option 3 If TV does not have a headphone or audio output jacks, you can place an optional microphone module directly in front of your TV loudspeaker to pick up the sound for transmission. Connect the other end of the microphone module to "TV Audio" at the rear of the transmitter base. Set the Selector to the MIC position. (This optional microphone module model TV-MIC ,item number TV6001 is available from a Serene Dealer).

PLACEMENT OF THE TRANSMITTER BASE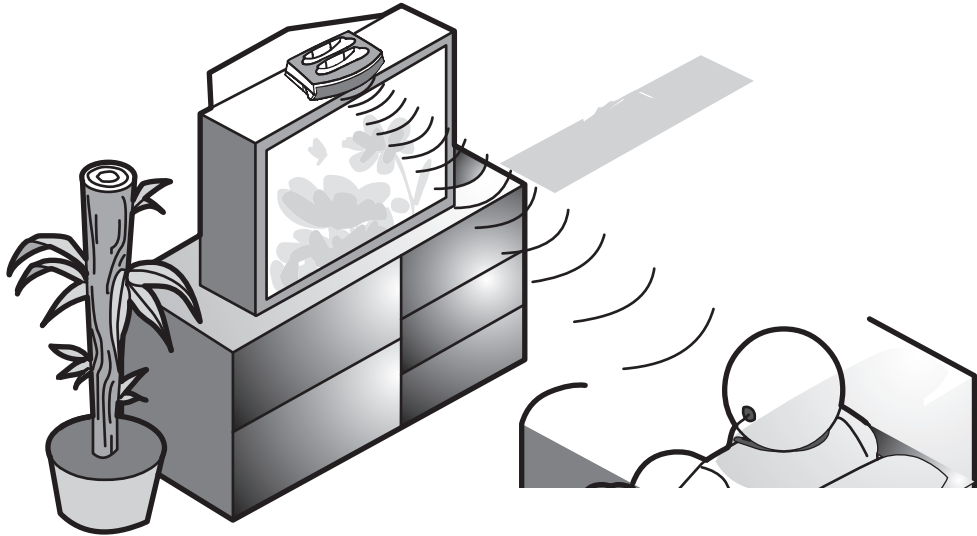


Place the transmitter on top of your TV or on your TV stand so that its front (line of sight) is pointed directly at the viewer. Make sure there is nothing in front of the receiver blocking the path. The powerful transmitter can cover any room size up to 30 ft × 30 ft.

USING THE TV-DIRECT RECEIVER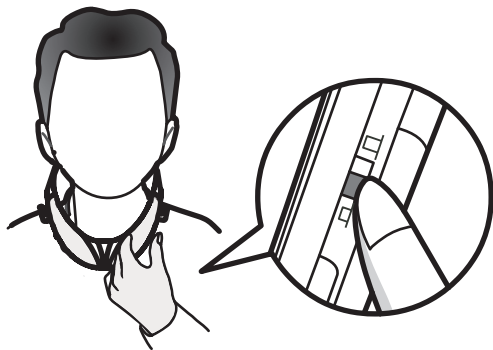

1. Set the volume to a low level (to protect your hearing from sudden loud sound) before putting on the ear buds.
2. Place the receiver gently on your shoulder and press the ON/OFF button to turn on the receiver, Its ON/OFF indicator will light up.
3. Adjust the sound level to a comfortable level.
4. Set the tone control to a position that enables you to hear and understand all the dialogue best.
5. The charging LED indicator in the receiver lights up as long as the receiver is placed in the transmitter for recharging. A fully charged battery provides about 30 hours of listening enjoyment.
6. The transmitter can charge two receivers at the same time. There is no limit on the number of receivers used with a transmitter.
7. To conserve battery power, the receiver automatically shuts off if no IR signal is detected for a period of 40-80 seconds.

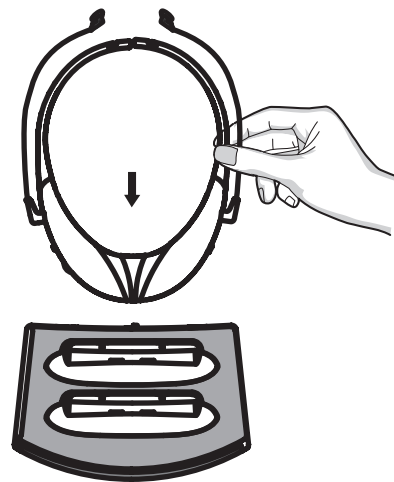SPECIFICATIONS

Selector switch	Stereo: For stereo input
	Mono: For mono input
	Mic: For microphone input
Modulation	FM stereo
Carrier frequencies	2.3/2.8 MHz
Range	Up to 30'
Frequency response	50Hz to 16 KHz
THD	Less than 3%
Receiver auto shut-off time	No IR signal is detected for approx. 40-80 seconds
Power supply (base)	12VDC / 400mA
Receiver battery life	Approximately 30 hours
Charging indicator:	Charge indicator is located in the receiver. It lights up when the receiver is placed on the charging bay. It takes 8 hours to fully charge the batteries.
Weight Transmitter	Approx 5.3 oz (150 g)
Receiver	Approx 3 oz (84 g)

TV-DIRECT

Optional Accessories

The following accessories enhance your TV listening experience. Please contact any dealer for availability.

1. Model TV-95RX (item TV5201):
Extra receiver for TV-95

2. Model TV-MIC (item TV-6001):
External microphone to attach to and to pick up sound from the TV speaker grille

3. Model TV-EBC (item TV6003):
3 pairs of replacement earbud covers for TV95 (one pair each Small, Medium, Large)

4. Model TV-TC (item Tv6005):
Single Silhouette style tele-coil coupler with 2.5 mm plug to work with T-coil hearing aid

5. Model TV-EB (item TV6002) :
Replacement ear-buds (one pair).

6. Model TV-NL (item TV6004) :
Light weight neck-loop for use with T-coiled hearing aids.

WARRANTY SERVICE INFORMATION

Your TV-Direct™ comes with a one-year limited warranty. Batteries and ear buds are warranted for 30 days only. We warrant during this warranty period from the date of purchase to the original consumer; that TV-Direct will be free from defects in materials and workmanship under intended home use. In the event that TV-Direct fails to function properly within one year of original purchase due to defects in materials or workmanship, return the unit (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to Serene Innovations. Serene Innovations will either repair or replace the unit (with a refurbished unit or unit of equal condition) and return it to you (using UPS/USPS ground shipping) at no cost to you if the unit is returned within 30 days of purchase. If the unit is returned after 30 days of purchase but within the warranty period, there will be a warranty handling charge of \$9.95 for each return.

This warranty does not apply to any product that has been accidentally damaged due to: abuse,

misuse, negligence, use on electrical frequency or voltage other than marked on product and/or described in manual. Defects or errors caused by unauthorized alterations, repairs, and/or tampering are also not covered by this warranty. This warranty gives you specific legal rights, and you may have other legal rights that vary from state to state. If a defect covered by this warranty should occur, promptly contact a Customer Care Representative by phone or log onto www.sereneinnovations.com to obtain a Return Authorization Number (RAN) and shipping instructions before shipping the product to us. Any shipment without a RAN will not be accepted and will be returned to you at your expense. Any authorized returned product must be accompanied with proof of purchase and a brief description of the problem. For out of warranty repair and service, please contact our Customer Care Department for directions.

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