
Customer Service: I-800-552-3368 www.clarityproducts.com

HOW TO CONNECT YOUR C4230 PHONE

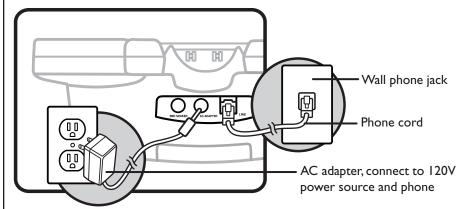

The **phone cord** plugs into the C4230 phone base as shown above. Plug the other end of the **phone cord** into a telephone wall jack.

Connect the **AC adapter** to the back of the base and a 120V wall outlet as shown above.

BASE FUNCTION KEYS

- **4 Speaker** Press to turn speaker On or Off.
- 5 Dial Pad Dial number calling.
- 6 Boost Press to add an extra level of amplification over the entire range of volume control.
- **7 Volume** Press to control the speaker and ringer volumes.
- **8 Power LED** LED will light up when the power is on.
- **9 In Use LED** LED will light up when the phone is in use.
- 10 Charge LED LED will light up when handset is in base charging.

ANSWERING MACHINE OPERATIONS

I On/Off


Press to turn answering machine **ON** or **OFF.**

2 Delete

Press to **Delete** a message.

3 Play

Press to Play message.

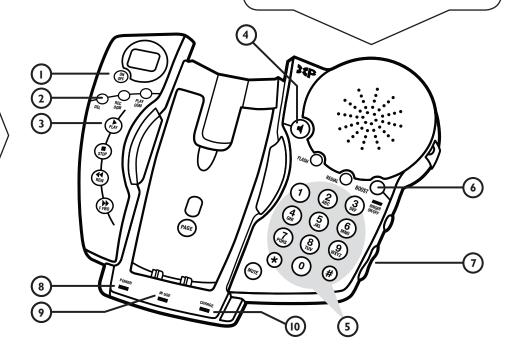

Customer Service: 1-800-552-3368 www.clarityproducts.com

KEY HANDSET OPERATIONS

I Talk

Press TALK button and dial number to make a call. To end call press TALK again.

2 Speaker

Press to activate handset speakerphone. Press again to switch back to earpiece.

3 Boost

When pressed, an extra level of amplification is added over the entire range of volume control. (Digital Clarity® PowerTM Technology)

4 Volume Control


The dial controls the volume level of the handset, headset or neckloop.

5 Tone

The C4230 has four (4) AUDIO TONE settings. Pressing the TONE button changes the style of amplification and allows you to customize your hearing experience. Hearing loss is specific to each individual; therefore, we suggest exploring each setting to find the one you find most comfortable.

6 Ringer On/Off

Switch to turn Handset Ringer On or Off.

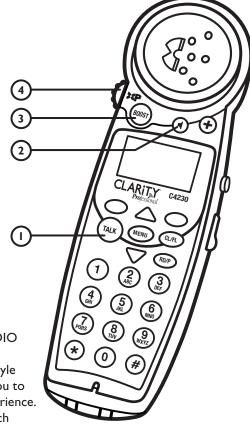

HOW TO DIAL OR RECEIVE A CALL

- I. Complete connection instructions shown on reverse side of this guide.
- 2. Install and charge battery as shown below.
- 3. Make sure the **power LED** is on at bottom of base.

HANDSET

- To receive a call pick up handset from base and press TALK to answer incoming call.
- 5. When placing a call press **TALK** and then dial the number.
- After completing the call press TALK again and return handset to base unit.

BASE


- 7. When receiving a call press **SPEAKER** button on base.
- 8. When placing a call press **SPEAKER** button then dial number from base.
- After completing the call press the SPEAKER button again.
- 10. Sound will reset to normal level on both handset and base unit after completing calls. Pressing the **BOOST** button on subsequent calls will return the volume to your set sound level.



Handset battery must charge 12 hours before use.

Battery

- Insert battery pack such that (+) and (-) on battery touch spring in battery compartment.
- Place handset on base to charge.
- Make sure the charge LED on base is illuminated.

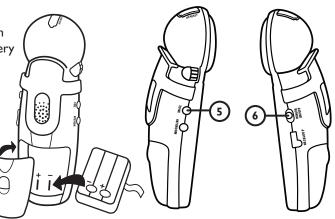



Handset and base combination