

Setup Guide

1 Unpack



A400E DECT 6.0
Amplified Cordless
Expandable Handset

Please see the *ClearSounds A400 DECT 6.0 Amplified Cordless Phone User's Guide* for additional instructions.



Telephone handset



A400E charger



AC power adapter assembly



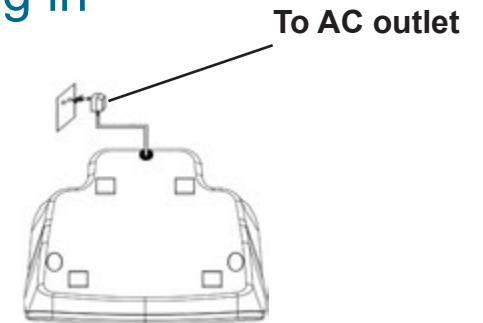
Three rechargeable AAA
NiMH batteries



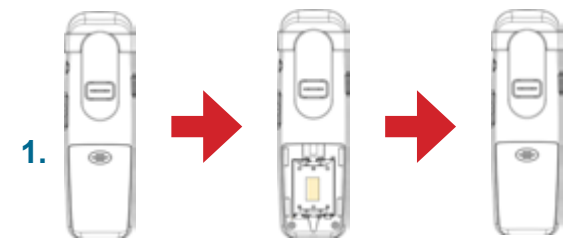
Carrying clip

2 Power

Plug in



Charge





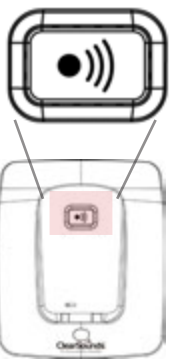

1.





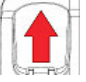


2.

Prior to use, charge for 24 hours.

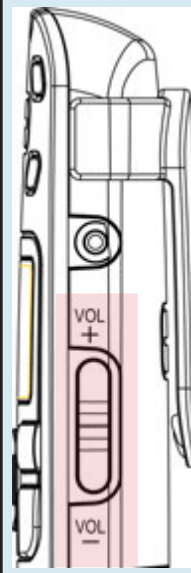
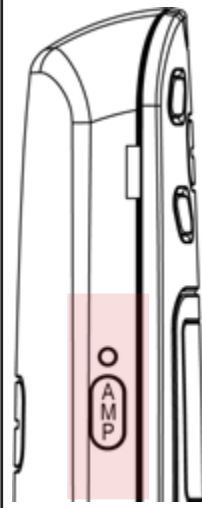

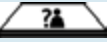
3 Register

| | |
|---|---|
|  | <p>1. On the unregistered A400E handset, press and hold  beneath Menu on the screen, until Press and hold base Page key appears, and then release the handset key.</p> |
|  | <p>2. On the A400 base to which you are registering, press and hold  for 10 seconds, and then release the key.</p> <p>When the handset is registered to the base, the handset number appears on the handset screen.</p> |

4 Call

| Place | Answer | End |
|--|--|--|
| <p>1. Dial .</p> <p>2. Press .</p> | <p>1.  Lift.</p> <p>2. Press .</p> | <p>1. Press .</p> |

5 Volume

| Volume | Amplify | Equalize |
|--|---|---|
|  <ul style="list-style-type: none"> • Press + or -. |  <ul style="list-style-type: none"> • Press to turn on or off. | <p>1. Press .</p> <p>2. Press  to select Treble, Flat or Bass.</p> <p>As you select, a graph appears on the screen.</p> |

ClearSounds 1-800-965-9043