

# Setup guide



## Connect 360 System



# 1 Unpack



QUATTRO amplified Bluetooth™ neckloop



QLINK Bluetooth™ stereo transmitter




RS062 SmartSound earbuds




USB cable



RCA cable

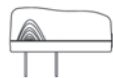

QUATTRO and QLINK AC chargers (two chargers)

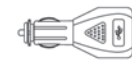

QUATTRO car charger




*QUATTRO Neckloop Accessory Operating Manual and QLINK Transmitter Operating Manual*  
Please refer to these guides for additional instructions.

# 2 Charge


Prior to use, charge components for 3 hours.

## QUATTRO neckloop

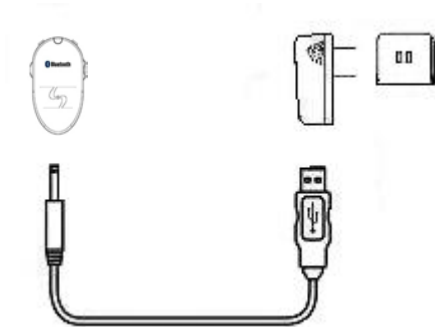

When fully charged, the blue light extinguishes.

## QLINK transmitter

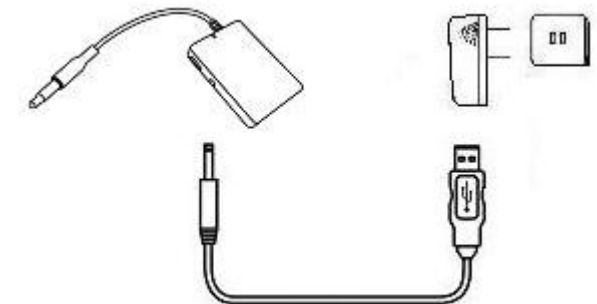


When fully charged, the red light extinguishes.

**Tip:** It is convenient to keep the QLINK plugged in and continuously charging.

# 3 Connect

You can use the Connect 360 System with televisions, computers, smartphones, cell phones and portable media players.








This setup guide describes using the system in two ways:

- With a television
- Simultaneously with a television and a cell phone or smartphone: If the phone rings while you listen to TV, you can press  on the neckloop to answer the

phone, and then press it again to end the call. During conversation, the TV mutes.


For connection information to the other devices, please refer to the enclosed directions for use.

## Connect the QLINK transmitter to a TV


 <p><b>OR</b> includes a 3.5 mm headset jack</p>	 <p>No 3.5 mm headset jack</p>
<p>1. Connect  to the 3.5 mm TV headset jack.</p> <p><b>Note:</b> When you try the 360 system, if the TV mutes and you prefer that it <i>not</i> mute, connect the QLINK transmitter using , instead (see right).</p>	<p>1. Connect  to the black jack on .</p> <p>2. Connect the  red and white connectors to the red and white audio-out jacks on the device that is connected to the TV cable (for example, it might be a cable box, a DVD player or the TV itself).</p>

# 4 Pair


1. Turn off both devices:

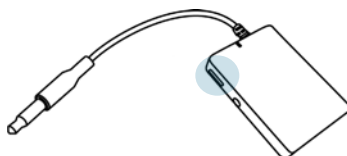

Press and hold until the red light flashes.

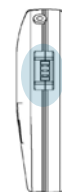

Press and hold until tones occur and the light is off.

**Note:** If you are simultaneously using a phone with the TV, turn off the phone's  **Bluetooth™**. (For instructions, see the phone user's guide).


- 2.





← 2 - 3 feet →

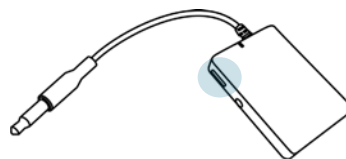


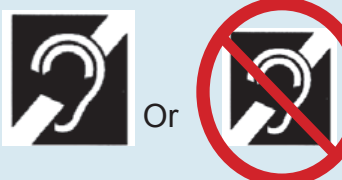
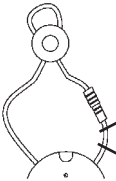
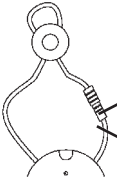



3. Press and hold until the lights flash blue and red:



When pairing occurs, the blue lights flash rapidly and the neckloop emits a tone.

**Note:** If you are using a phone, turn on the phone's  **Bluetooth™** and pair it with . (For instructions, see the phone user's guide).

# 5 Volume

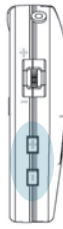 <p><b>T-coil</b></p>	 <p>Or</p>
<p>1. Switch your hearing aids to the "T" mode.</p>	<p>1.  Two clicks</p>
<p>2.  Two clicks</p>	<p>2.  Two clicks</p>
<p>3. Volume: </p>	<p>3. Volume: </p>



**ClearSounds 1-800-965-9043**