



### Step 3.

Connect the 3.5mm Stereo Audio Cable from the Geemarc V2T-10 to the Android Tablet. Connect the Tablet Power Cable from the Android Tablet charging port to the Tablet Power Supply. Then connect the Tablet Power Supply to an Electrical Outlet.

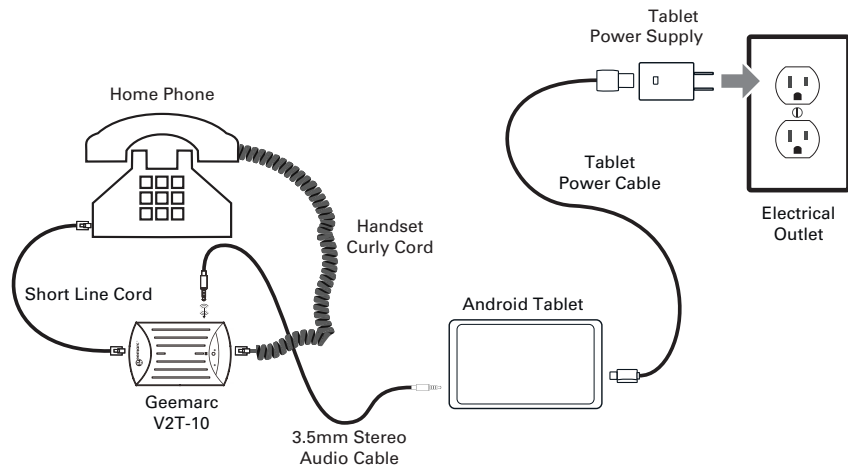


### Step 4.

Turn on the Geemarc V2T-10 and set to "Auto" position.

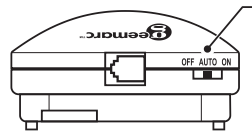

### Step 5.

Turn on the tablet by holding down the On/Off button for 5 seconds.

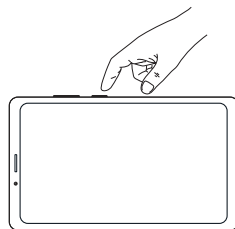

### Step 6.

Start using your Teltex GLT Tablet to caption all of your calls.

**For Setup Videos Visit:**

<https://www.iaccessibility.com/videos/deaf-hard-of-hearing/>


# Teltex GLT Captioning Tablet with V2T-10 Quick Setup Guide






### What's in the Box:

- (A) Geemarc V2T-10
- (B) 9-Volt Battery
- (C) Short line cord
- (D) 3.5mm Stereo Audio Cable
- (E) Android Tablet
- (F) Tablet Power Supply
- (G) Tablet Power Cable
- (H) Mini WiFi Router
- (i) WiFi Power Supply
- (J) WiFi Power Cable
- (K) Ethernet Cable
- (L) Tablet Stand

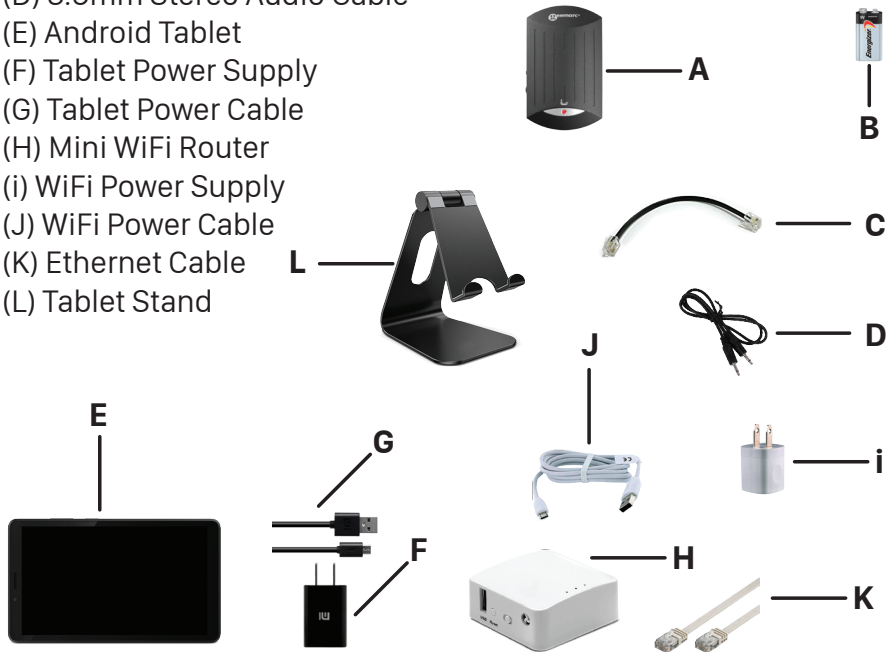

### Step 1.

Install 9-volt battery into V2T-10 unit and connect the V2T-10 unit to telephone as shown below.

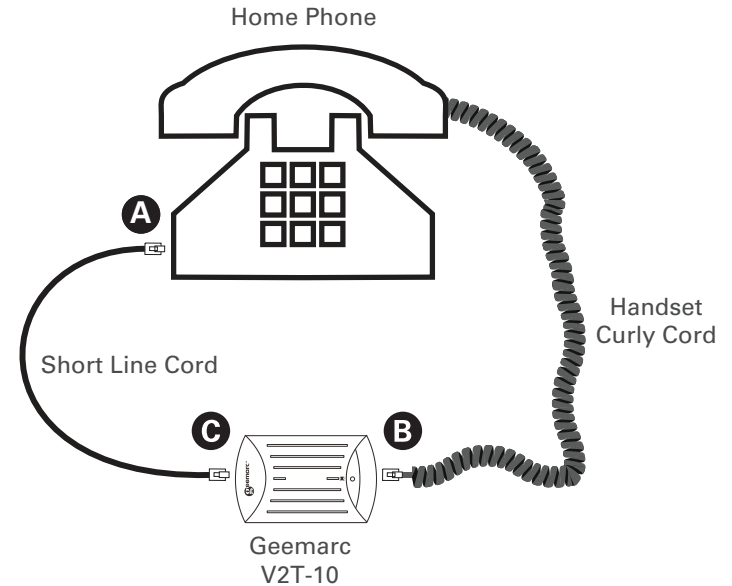

### Step 2.

Connect the WiFi Power Cable to the WiFi Power Supply, then connect it to an Electrical Outlet. Connect the WiFi Power Cable to the Mini WiFi Router **Power** jack.

Next, connect the Ethernet Cable to the **WAN** port on Mini WiFi Router and connect the other end of the Ethernet Cable to your Internet Router (not included).

