12082 Western Avenue, Garden Grove, CA 92841
 VOICE (800) 874-3005 TTY (800) 772-2889
 FAX (714) 897-4703
 ameriphones@plantronics.com www.ameriphone.com

AMERIPHONE
OPERATING INSTRUCTIONS

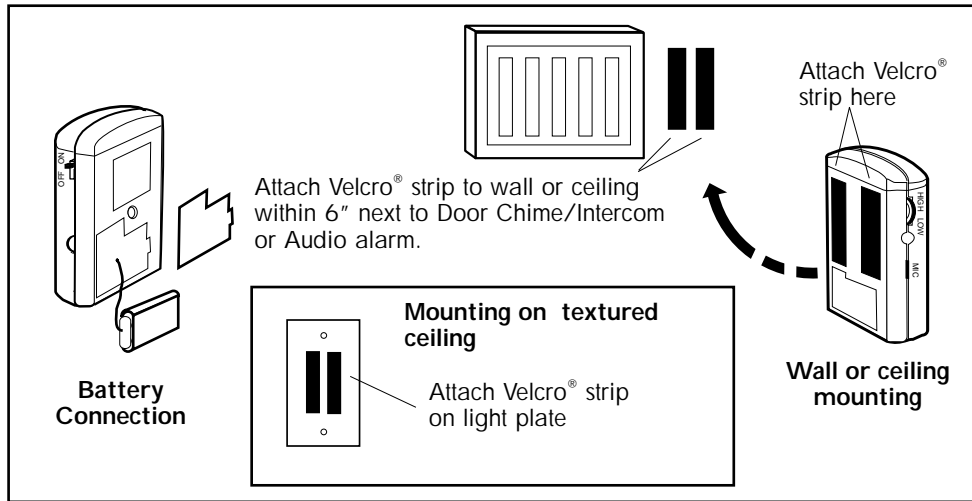
Works with ALERTMASTER® Product Series
Door Announcer (AM-DX)
Baby Sound Monitor (AM-BX)
Audio Alarm (AM-AX)
ALERTMASTER® Transmitter

LIMITED WARRANTY

Ameriphone warrants the AM-AX/AM-BX against any defect in workmanship or material, for a period of one year from the date of purchase. If the product is defective and returned within 30 days of purchase, the unit will be replaced at the place of purchase at no charge. Ameriphone will repair or replace it as it is returned after 30 days but within one year. Ameriphone will repair or replace it as long as the original design. Repair or replacement for either party will be warranted for either 90 days or the remaining time on the warranty period, whichever is longer. When in need of warranty repair, please call our customer service department at (800) 874-3005 (voice), (800) 772-2889 (TTY). They will help you process your return shipment. You must prepay all shipping costs. Implied warranties, including those of fitness for a particular purpose and merchantability, are hereby disclaimed. Some states do not allow limitations on how long an implied warranty lasts or the exclusion of incidental or consequential damages, so the above exclusions or limitations may not apply to you. The warrantor does not cover defects resulting from accidents, damage while in transit to our service location, alterations, unauthorized repair, failure to follow instructions, mis-use, use outside the United States or Canada, fire, flood and acts of God. We request you save the original package materials in the event that you need to ship your AM-AX/AM-BX back. When shipping for warranty repair, include your name, address, phone number, proof of date of purchase and a description of the problem. After repair of the product, we will ship it back to you at no cost within the United States.

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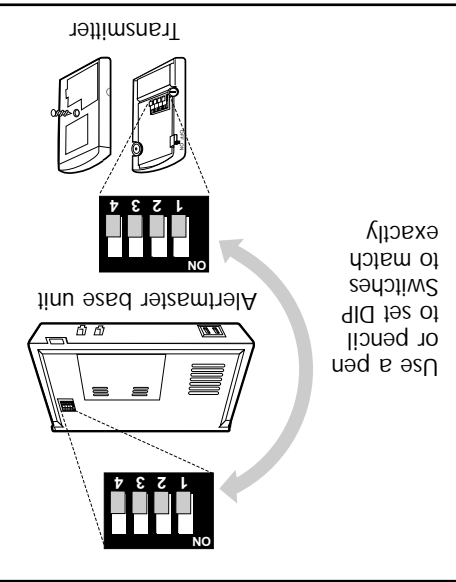
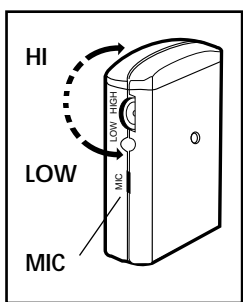
PLACEMENT

- Place the AM-BX with MIC pointing toward and as close to the baby as possible, but no more than 5 feet away.
- Place the AM-AX and AM-DX with MIC pointing toward and within 6 inches from the audio alarm device or door chime/intercom respectively.

For installation on textured ceilings, screw a standard light switch cover, readily available from your local hardware store, on ceiling and apply the velcro® strips to the plate instead of directly on ceiling.

BATTERY AND OPERATION TESTS

Check or change the battery every six months to make sure it is properly charged.

AM-AX Press the Test button on your audio alarm device. Check that the Alarm function comes on at the Alertmaster console after 12 seconds.

AM-BX Make loud noise continuously for 5 seconds or longer. Check that the Sound function comes on the Alertmaster console.

AM-DX Press the intercom or door chime button. Check that the Door function comes on at the Alertmaster console.

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
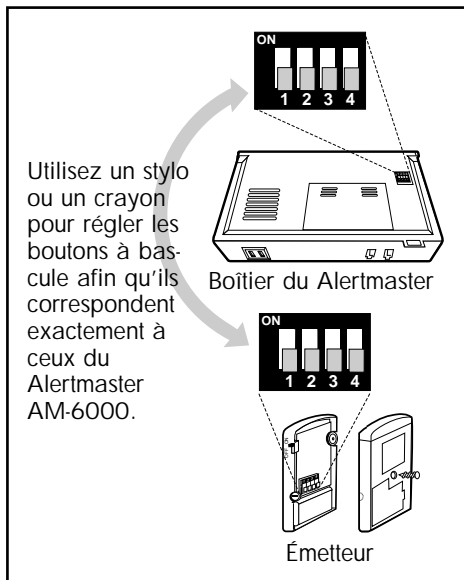
The Alertmaster System uses special radio signals to communicate between the console unit and all its accessories. For all accessories to activate the base unit, they all have to be on the same "channel". A "DIP Switch" is used for channel selection. Your system and all the accessories have been pre-set in the factory to a standard channel setting. Under normal operating conditions, there is no adjustment necessary. If your system encounters interference and repeatedly activates by a system nearby in your neighborhood. Change channel switch setting in the console. All settings must be identical for the system to work.

- Locate the DIP Switches on the back of the AM-6000 or the back of the AM-7000 console.
- Change the switch setting by turning all individual switches to a different position.
- Unscrew housing and locate DIP Switch inside of the transmitter.
- Change the switch settings on each of your other accessories (doorbell, Remote Receiver, etc.) to match the new switch setting in the console. All settings must be identical for the system to work.

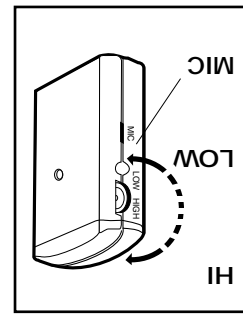CHANNEL SETTING

All accessories have been pre-set to their optimum sensitivity in the factory. But, for best results and to suit your application, follow these steps to adjust their sensitivity.

SENSITIVITY ADJUSTMENT

IN CASE OF DIFFICULTY

If any problem should occur during operation of your Alertmaster AM-AX, AM-BX, or AM-DX accessory, review the list below and your Alertmaster AM-6000 or AM-7000 Operating Instructions. Should the difficulty persist, contact AMERIPHONE or your authorized dealer for assistance.

SYMPTOM	CAUSE & REMEDY
System does not respond when Alertmaster AM-accessory is tested.	Adjust the sensitivity. Or battery needs to be replaced.
Alertmaster console does not respond when Alertmaster AM-accessory is tested.	The accessory may not be at the right "Channel Setting". See "Channel Setting" section.
System activates erratically	If the System picks up other wireless signals, change channel setting on all components. See "Channel Setting".

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