PowerTel 601

User Guide


Getting Started	5	Help and Support.....	10
Registration Key Guide	7	Troubleshooting Guide	10
Using the Wrist Shaker	8	Maintenance and Warranty	12

FCC Compliance

The FCC requires that you will be advised of certain requirements involving the use of this telephone.

1. This equipment complies with Part 68 of the FCC rules. A label on the base unit of this equipment contains, among other information, the ACTA Registration Number (US:AAAEQ#TXXXX) and the Ringer Equivalence Number (REN) for this equipment. You must, upon request, provide this information to your telephone company.
 2. An applicable certification jacks Universal Service Order Codes (USOC) for the equipment is provided (i.e., RJ11C) in the packaging with each piece of approved terminal equipment.
 3. A plug and jack used to connect this equipment to the premises wiring and telephone network must comply with the applicable FCC Part 68 rules and requirements adopted by the ACTA. A compliant telephone cord and modular plug is provided with this product. It is designed to be connected to a compatible modular jack that is also compliant. See installation instructions for details.
 4. The REN is useful to determine the quantity of devices you may connect to your telephone line and still have all of those devices ring when your number is called. In most, but not all areas, the sum of the REN of all devices should not exceed five (5.0). To be certain of the number of devices you may connect to your line, as determined by the REN, you should call your local telephone company to determine the maximum REN for your calling area. NOTICE: If your home has specially wired alarm equipment connected to the telephone line, ensure the installation of this equipment does not disable your alarm equipment. If you have questions about what will disable alarm equipment, consult your telephone company or a qualified installer.
 5. If your telephone causes harm to the telephone network, the telephone company may discontinue your service temporarily. If possible, they will notify you in advance. But if advance notice is not practical, you will be notified as soon as possible. You will be advised of your right to file a complaint with the FCC.
 6. Your telephone company may make changes to its facilities, equipment, operations or procedures that could affect the proper operation of your equipment. If they do, you will be given advance notice so as to give you an opportunity to maintain uninterrupted service.
 7. If you experience trouble with the telephone, please contact the Amplicom Service Center at 1-866-AMPLICOM (267-5426) for repair/warranty information. The telephone company may ask you to disconnect this equipment from the network until the problem have been corrected or you are sure that the equipment is not malfunctioning.
 8. This equipment may not be used on coin service provided by the telephone company. Connection to party lines is subject to state tariffs. (Contact your state public utility commission or corporation commission for information.)
- This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Privacy of communications may not be ensured when using this phone. Changes or modifications not expressly approved in writing by Amplicom may void the user's authority to operate this equipment.
- Some cordless phones operate at frequencies that may cause interference to nearby TVs and VCRs; to minimize or prevent such interference, the base of the cordless phone should not be placed near or on top of a TV or VCR, and, if interference is experienced, moving the cordless telephone farther away from the TV or VCR will often reduce or eliminate the interference. NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures: Reorient or relocate the receiving antenna, increase the separation between the equipment and receiver, connect the equipment into an outlet on a circuit different from that to which the receiver is connected or consult the dealer or an experienced radio/TV technician for help.
- For body worn operation, this handset has been tested and meets the FCC RF exposure guidelines when used with the Amplicom accessories supplied or designated for this product. Use of other accessories may not ensure compliance with FCC RF exposure guidelines.

PowerTel 601 Wrist shaker

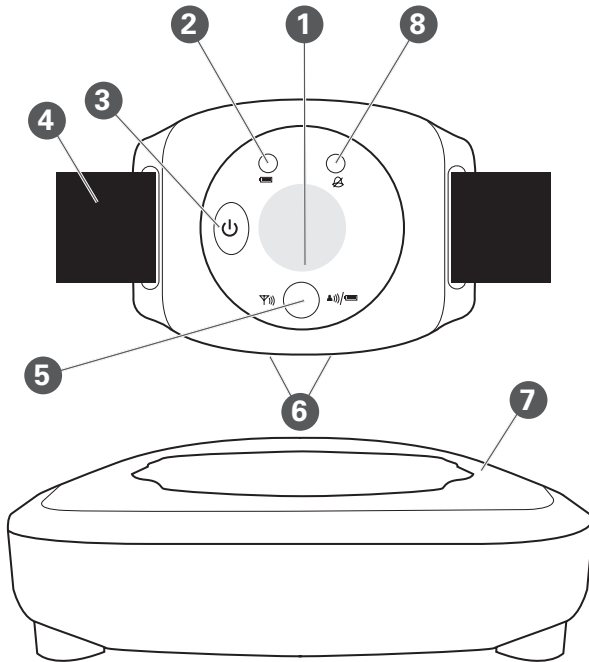- ① Emergency button
Note: Can only be used with the PowerTel 680 telephone
- ② Battery charging red light
- ③ Power on/off
- ④ Wrist strap
- ⑤ Registration mode/
Out of range/Low battery 2
color green or red light
- ⑥ Charging contacts
- ⑦ Charging unit
- ⑧ Incoming call green light

Safety information - SAVE THESE INSTRUCTIONS

Unauthorized modification or reconstruction is not permitted. **Do not** open the device and attempt any repair work yourself.

Charger unit location

Locate the charger unit:

- On a dry, flat and stable surface
- Where the power cable will not be a trip hazard
- Where it is not obstructed by furniture
- At least three feet from other electrical appliances
- Away from smoke, dust, vibration, chemicals, moisture, heat or direct sunlight.

Power failure – What to do

This product cannot be used in the event of a power failure. Make sure alternative arrangements are made in place of emergency calls.

Medical equipment

Do not use the phone near emergency or intensive care medical equipment, or during an electrical storm. If you have a pacemaker check with a medical expert before using this product.

Important: Use only power supply model S008CU1200067 supplied by Ten Pao Industrial Co., Ltd. for Base unit and charger adapter S005IU0550030 (optional).

Getting Started — Included Equipment

- 1 wrist shaker with a wrist strap attached
- 1 charger unit with power adapter plug attached

Set up


1. Position the charger unit

2. Plug in charger unit

Plug the charger unit into power supply and switch the power on.

3. Place the wrist shaker in charger unit

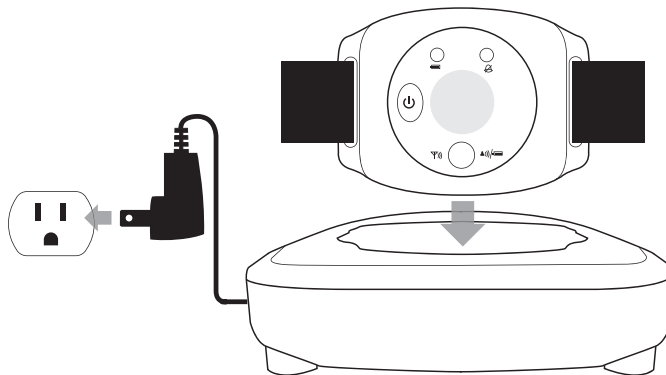Place wrist shaker on its charger unit with the two connection points facing down. A red light on the wrist shaker indicates it is charging properly. Charge the wrist shaker for 10 hours before use.

Note: When fully charged, the wrist shaker's battery will last five days. The larger red light will flash slowly when the battery is low and needs to charge for 10 hours.

4. Register the wrist shaker to the PowerTel telephone

Important: If the wrist shaker was purchased separately from a PowerTel phone, it will need to be registered prior to use.

- i. Turn on the wrist shaker by pressing and holding the  button for five seconds.
- ii. Press and hold the  button again for 12 seconds, until the registration light flashes green quickly. The wrist shaker will be in registration mode for about 1 minute.
- iii. Activate registration mode on the PowerTel phone.

For a PowerTel 500 or PowerTel 580 phone:

Press and hold the registration key  for about ten seconds until the charging light flashes.


For a PowerTel 580 Combo:

Press and hold  until **Register** on the display.

- iv. The wrist shaker is registered and ready to use when the registration light is a steady green (i.e. it isn't flashing).

Note: The registration light will flash green and red if your wrist shaker isn't registered or is out of range. Move closer to the PowerTel telephone to ensure you are within range.

Registration key guide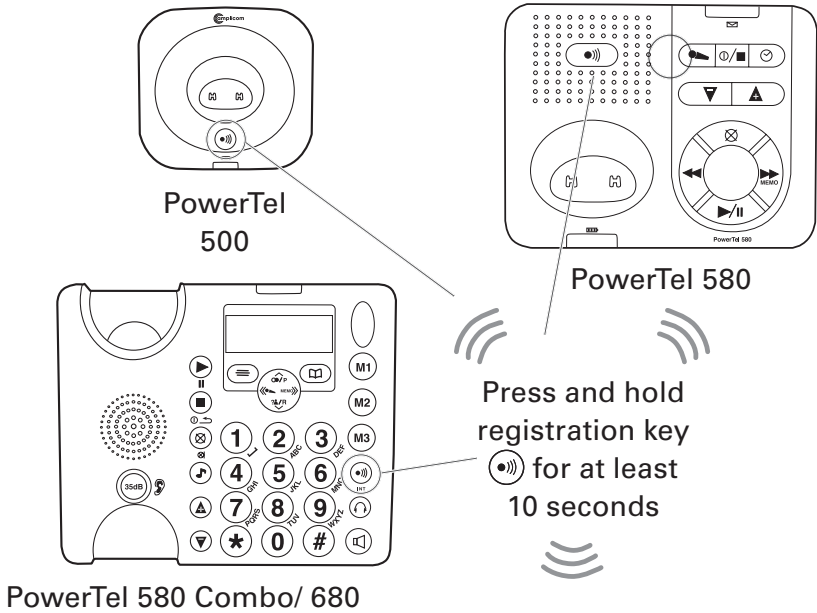


Your PowerTel 601 wrist shaker is now ready for use!

Using the wrist shaker

Switch on

Press and hold the  button for about five seconds. The green light with the warning triangle will illuminate.

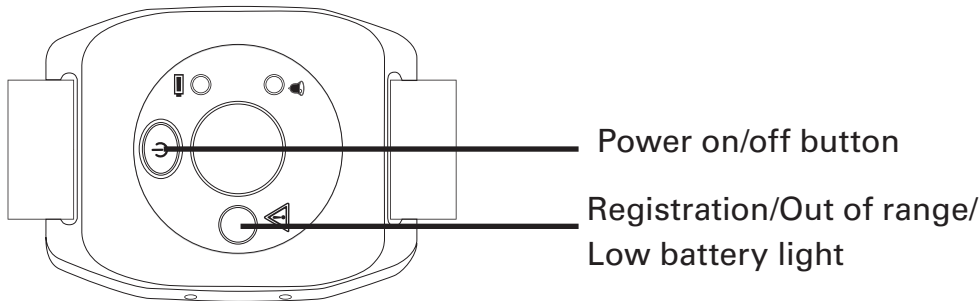Switch off

Press and hold the  button for about five seconds. The green light will turn off.

Wrist strap

The wrist strap comes pre-attached to the wrist shaker. To reattach it or adjust it:

Thread the wrist strap through the slot on one side of the wrist shaker (front to back), around the back and then through the other slot (back to front), just like a wrist watch.

Tighten the Velcro fastener until comfortable.

Make sure the wrist shaker is in range

If the wrist shaker is not in range, it won't work. The registration light will flash green and red when the shaker is out of range. Move closer to the PowerTel phone base.

Battery low

If the battery is low, the low battery light will turn red and will flash. Return the wrist shaker to charger base.

Incoming calls

When an incoming call is received, the wrist shaker vibrates and a bell symbol flashes green on its display.

Answer and end a call in the usual way using the PowerTel phone.

Help and support

Troubleshooting guide

My wrist shaker doesn't vibrate when I have an incoming call.

Make sure you've registered your wrist shaker. If you purchased your wrist shaker separate from your phone, you need to register it. Please see Step 4 of Getting Started on page 6.

If you have registered your wrist shaker and it still doesn't vibrate with an incoming call, you may have moved out of range of your phone. Move closer to your phone.

My display is flashing red.

The battery is low. Recharge your wrist shaker by placing it on your charger unit. To fully charge, it will take about ten hours.

Energy efficient power adapter

The included power adapter meets the eco-design requirements of the European Union (Directive 2005/32/EG).

Disposal

The  symbol on this product indicates that electrical and electronic apparatus and  battery pack must be disposed of **separately** from domestic waste at suitable collection points provided by the public waste authorities.

Maintenance and Warranty

Maintenance

- Please clean your equipment's surfaces with a soft, lint-free cloth.
- Never use cleaning agents or solvents.

Warranty

AMPLICOM equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life. The terms of the warranty do not apply where the cause of the equipment malfunction is the fault of the telephone network operator or any interposed private branch extension system. The terms of the warranty do not apply to the rechargeable battery pack or power packs used in the products. The period of warranty is 24 months from the date of purchase.

All deficiencies resulting from material of production faults which occur during the period of warranty will be eliminated free of charge. Rights to claims under the terms of warranty are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of warranty.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become

our property.

Rights to compensation in the case of damage are excluded where there is no evidence of intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of warranty, please return to the sales outlet in which you purchased the AMPLICOM equipment together with the purchase receipt. All rights to claims under the terms of warranty in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of warranty can no longer be asserted.

Declaration of conformity

The ETL Listed Mark is an alternative to the CSA and UL marks. ETL Testing Laboratories owned by Intertek Testing Services (ITS) is recognized by OSHA as a Nationally Recognized Testing Laboratory (NRTL). ITS tests products according to nearly 200 safety and performance standards. The ETL Listed Mark is accepted throughout the United States when denoting compliance with nationally recognized standards such as ANSI, IEC, UL, and CSA.

This certification mark indicates that the product has been tested to and has met the minimum requirements of a widely recognized (consensus) U.S. product safety standard, that the manufacturing site has been audited, and that the applicant has agreed to a program of periodic factory follow-up inspections to verify continued conformance.

THIS PAGE LEFT INTENTIONALLY BLANK

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Distributed by: Amplicom USA LLC
Distribuido por: Amplicom USA LLC