

Distributed By:

Pocket  
**listenor**

## User's Guide



**Hands-free personal  
assistive listening device**

*For use with Telecoil-equipped hearing instruments*

Pocket  
**listenor**

[www.conversorproducts.com](http://www.conversorproducts.com)

### Intended Use:

The *Pocket Listener* is intended to be used as an Assistive Listening Device by persons who are hard of hearing.

### EC-Declaration of Conformity



We, Conversor Limited of

The Lansbury Estate,  
102 Lower Guildford Road,  
Knaphill, Woking, Surrey  
GU21 2EP  
United Kingdom

Declare that the product:

The Pocket Listener Assistive Listening Device is in Conformity with Council Directive 204/108/EC (EMC DIRECTIVE)

EN 61000-6-1 : 2001 Generic Standards – Immunity for residential, commercial and light-industrial environments

EN 61000-6-3 : 2001 Generic Standards – Emission Standard for residential, commercial and light-industrial environments.

EN 61000-3-2 : 2001 Electromagnetic Compatibility

EN 61000-3-3 : 1995 Electromagnetic Compatibility

EN61000-4-3:2002 Electromagnetic compatibility. The Pocket Listener may be subject to type B degradation of performance.

Conversor Limited also declares the conformity of above mentioned Product with the actual required

safety standards in accordance with LVD 73/23 EEC.

EN 60950-1 : 2001 Safety

EN 60950-1 : 2003 Safety

Conversor Limited

February 2007.

#### Manufacturer's Warranty:

The manufacturer of this product issues a one-year warranty to the original consumer of this product. This warranty will go into effect upon the date of original purchase of this equipment, and will remain in effect for one calendar year from that date, and will be subject to the manufacturer's warranty terms and conditions a copy of which can be obtained from any authorised Distributor of this Product.

The Pocket Listener may be subject to temporary Disturbance from radio frequency transmitters, e.g. mobile phones, and there may result a temporary loss of function or degradation of performance, which ceases after the disturbance without user intervention required.

#### FCC ID: POCKET LISTENOR – UFA01PKL

This device complies with Part 15 of the FCC Rules Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.



NI 4279



MANUFACTURED IN THE EU FOR CONVERSOR LIMITED

The Lansbury Estate, Lower Guildford Road, Knaphill, Woking,  
Surrey GU21 2EP, United Kingdom

#### Conversor Limited

Free Phone: (0)800 298 9982 (UK Only)

Tel: +44 (0)870 066 3499

Fax: +44 (0)870 066 3669

info@conversorproducts.com

[www.conversorproducts.com](http://www.conversorproducts.com)

HBK01PKL\_INT

## Optional Accessories

The Pocket Listener can also be connected to a television, DVD and video players, or Cable TV boxes using the optional TV Scart Adaptor Cable, which is sold separately.



*TV SCART Adaptor Cable*

## Contents

<b>Easy 1-2-3 Operation</b> .....	<b>2</b>
How to Charge the Pocket Listener .....	2
<b>Connecting to Audio/Visual Equipment</b> .....	<b>3</b>
<b>Optional Accessories</b> .....	<b>4</b>

## Easy 1-2-3 Operation

1. Select the Telecoil ('T') setting on your hearing instrument(s).
2. Place the Pocket Listener around your neck and turn it on using the ON/OFF switch at the bottom of the unit.
3. Adjust the volume to a comfortable level by using the adjustment wheel on the side of the unit.

### How to Charge the Pocket Listener

The Pocket Listener is equipped with a rechargeable battery that provides 6 to 8 hours of continuous use.

A bright green LED shines continuously when the battery is fully charged. The LED gets dimmer as the battery is used. When there is no light, the Pocket Listener needs recharging.

### To re-charge the battery, please follow these simple instructions:

1. Connect the jack of the one-piece charging unit (supplied) to the charging socket located on the side of the Pocket Listener.
2. Connect the charging unit to an electrical outlet. (Please refer to the instructions supplied with the Charger.)
3. The recharging cycle takes up to 2 hours when the battery is completely drained.



## Connecting to Audio/Visual Equipment

### For use with MP3 Players, Hi-Fi Equipment/Stereos, and Laptops:

1. Connect the Audio Jack-to-Jack Cable (supplied) directly into the Audio Input Socket located at the top of the Pocket Listener using the red coloured plug (as shown in the pictures to the right).
2. Plug the other end directly into the headphone/audio output socket on your music device or laptop.
3. Turn on the Pocket Listener using the ON/OFF switch located on the bottom of the unit.
4. Adjust the volume to a comfortable level by using the volume wheel on the side.



### For use with Television, DVD and Video Players, or Cable TV boxes:

1. Connect the TV Scart Adaptor Cable (sold separately) into the Audio Input Socket located at the top of the Pocket Listener (as shown in the above pictures).
2. Please refer to the user guide supplied with this accessory for instructions on how to connect it to the audio/visual device.