



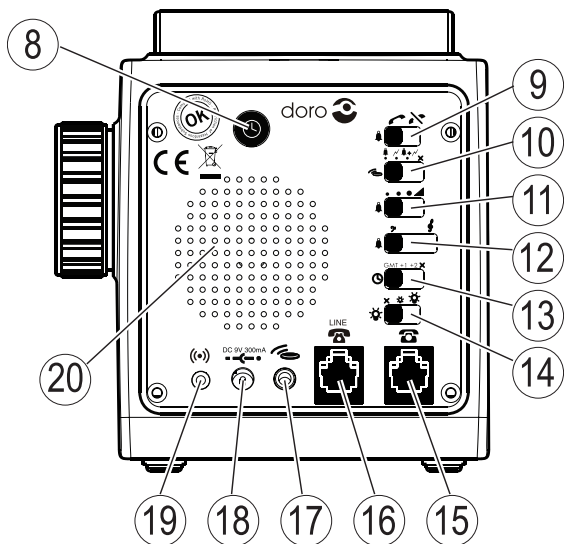

HearPlus  
333cl

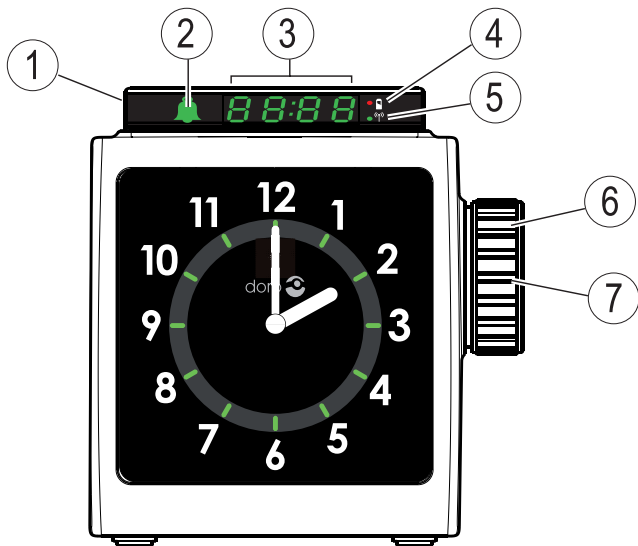

## English


- |                           |                           |
|---------------------------|---------------------------|
| 1. Alarm switch (display) | 13. Time zone             |
| 2. Alarm indicator        | 14. Brightness            |
| 3. Alarm time             | 15. Phone socket          |
| 4. Battery indicator      | 16. Line socket           |
| 5. Radio signal indicator | 17. Shaker socket         |
| 6. Jog wheel              | 18. Mains adapter socket  |
| 7. Jog wheel button       | 19. External alarm socket |
| 8. Time set               | 20. Loudspeaker           |
| 9. Phone alarm on/off     |                           |
| 10. Alarm type            |                           |
| 11. Alarm volume          |                           |
| 12. Alarm tone            |                           |

## Svenska

- |                                 |                            |
|---------------------------------|----------------------------|
| 1. Alarmomkopplare<br>(display) | 11. Alarmvolym             |
| 2. Alarmstatus                  | 12. Alarmton               |
| 3. Alarmtid                     | 13. Tidszon                |
| 4. Batteristatus                | 14. Ljusstyrka             |
| 5. Radiosignalsstatus           | 15. Telefonuttag           |
| 6. Inställningsratt             | 16. Linjeuttag             |
| 7. Knapp på<br>inställningsratt | 17. Shakeruttag            |
| 8. Tidsinställning              | 18. Uttag för strömadapter |
| 9. Telefonalarm av/på           | 19. Externt alarmuttag     |
| 10. Alarmtyp                    | 20. Högtalare              |

# Installation

## Connecting

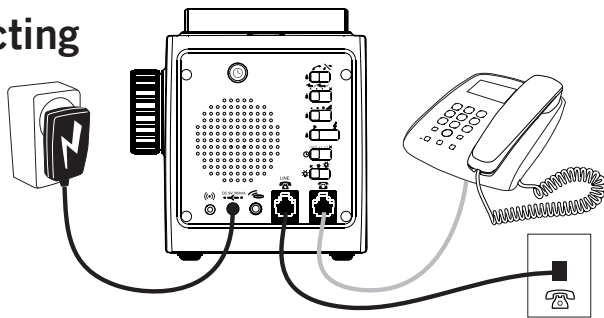

### **Note!**

*Set the time zone according to Settings/Timezone before connecting the power.*

Connect the mains adapter and, if applicable, phone line cords. A plug kit (not included) may be needed.

## Batteries

Remove the cover on the underside and install 4 AA 1.5V Alkaline batteries (not included) in accordance with the markings. The batteries are used for backup in case of power failure.

The clock can continue to operate for approximately 1 year on batteries. Backlight and display will automatically turn off when the clock is running on batteries only.

### **Note!**

*The pillow shaker and flash will not work during power failure.*

## Operation

### Setting the time

After connecting the power, the time is adjusted automatically by an RCC signal. The “P” indicator will light up on the display. If no RCC signal has been received within 20 minutes, the time should be set manually. The clock will retry automatic time adjustment at night when there is less radio interference.


The automatic time setting can be adjusted for the different time zones in Europe, see *Settings/Time zone*.

#### **Note!**


*Automatic time adjustment may not be available at all locations due to obstructions of the radio signal. Place the unit at least 2 meters away from a TV, computer, air conditioner or other household electrical appliances.*


***The optimal location is near a window, preferably facing Frankfurt/Germany.***

## Automatic time setting

1. Press and hold  until  $\text{rCC}$  is displayed to update the automatic time adjustment.

## Manual time setting

It is not possible to set the time manually while the clock is searching for an RCC signal, please wait until the  indicator is off.

1. Press . The display will light up and the hours will blink on the display.
2. Turn the jog wheel to set the hours and press the jog wheel button to confirm. The minutes will blink on the display.
3. Turn the jog wheel to set the minutes and press the jog wheel button to confirm.

### **Note!**


*The adjustment of the analog clock can take up to 30 minutes to complete. During the adjustment, the analog clock will not show the correct time. Do not disconnect the power during adjustment since this could damage the clock.*

## Setting the alarm time

1. Press the jog wheel button. The display will light up and the hours and the 🛎 symbol will blink.
2. Turn the jog wheel to set the hours and press the jog wheel button to confirm. The minutes will blink.
3. Turn the jog wheel to set the minutes and press the jog wheel button to confirm.

## Turning on the alarm

Toggle the alarm switch **forward** to turn on the alarm. The 🛎 symbol and the display will light up.

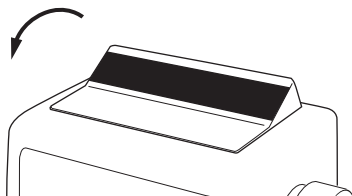


### **Note!**

*The display will time out after 10 seconds after disconnecting the mains adapter. Press the alarm switch to turn the display on again. The alarm time will not show while the clock is searching for an RCC signal.*

## Turning off the alarm

To turn off the alarm, toggle the alarm switch **backward**.

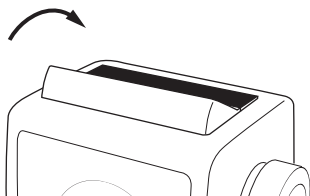


### **Note!**

*The alarm turns off automatically after 9 minutes.*

## Snooze


When the alarm is heard, press the alarm switch. The alarm will be repeated every 9 minutes until the alarm is turned off.

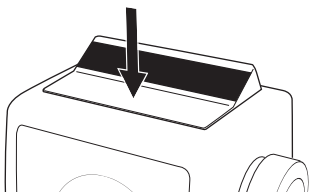




## Additional functions

### External alarm

An external alarm device such as Doro CarePlus or Doro SeePlus 314ci can be connected to the  socket.

If the alarm is not turned off or snoozed within 9 minutes, the external alarm will be triggered.

### Pillow shaker


A pillow shaker such as Doro HearPlus 55v can be connected to the  socket. See *Settings/Alarm mode* on how to enable/disable the shaker.

### Phone alarm

If the clock is connected to the phone line, incoming calls will trigger the alarm. See *Settings/Phone alarm* on how to enable/disable the phone alarm function. The sound and flash signals are determined by the *Alarm mode* setting when the phone alarm is ON.

# Settings

## Phone alarm

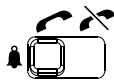


	Phone alarm ON
	Phone alarm OFF

## Alarm mode





×	Shaker only (if connected)
	Sound + Flash (+ Shaker if connected)
	Flash (+ Shaker if connected)
	Sound (+ Shaker if connected)
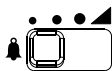
## Alarm volume



●	Low
●	High
●	Very High
	Gradually increasing alarm volume.

## Alarm tone

	Low frequency
	High frequency

## Time zone

The radio controlled time setting (RCC) can be adjusted for time zone differences.




<b>GMT</b>	GMT (London, Dublin...)
<b>+1</b>	GMT + 1 hour (Stockholm, Berlin, Paris...)
<b>+2</b>	GMT + 2 hours (Helsinki, Athens...)
<b>X</b>	Automatic time adjustment disabled. Time <b>must</b> be set manually.

### **Note!**

*The automatic adjustment (RCC) can take up to 30 minutes to complete. During the adjustment, the analog clock will not show the correct time.*

## Backlighting

The backlighting brightness setting also affects the LED display.

	No backlighting. Pressing the alarm switch will turn on the backlighting for 10 seconds.
	Dimmed backlighting. Pressing the alarm switch will change to bright backlighting for 10 seconds.
	Bright backlighting.

If the mains adapter is disconnected, backlighting is automatically turned off to save battery power. Press the alarm switch to turn on the lights for 10 seconds.

## Care and maintenance


Your unit is a technically advanced product and should be treated with the greatest care. Negligence may void the warranty.

- Protect the unit from moisture. Rain/snowfall, moisture and all types of liquid can contain substances that corrode the electronic circuits. If the unit gets wet, you should remove the mains power and the batteries and allow the unit to dry completely before you replace it.
- Do not use or keep the unit in dusty, dirty environments. The unit's moving parts and electronic components can be damaged.
- Do not keep the unit in warm places. High temperatures can reduce the lifespan for electronic equipment, damage batteries and distort or melt certain plastics.
- Do not keep the unit in cold places. When the unit warms up to normal temperature, condensation can form on the inside which can damage the electronic circuits.
- Do not try to open the unit in any other way than that which is indicated here.
- Do not drop the unit. Do not knock or shake it either. If it is treated roughly the circuits and precision mechanics can be broken.
- Do not use strong chemicals to clean the unit.

The advice above applies to the unit, battery, mains adapter and other accessories. If the unit is not working as it should, please contact the place of purchase for service. Don't forget the receipt or a copy of the invoice.

## Troubleshooting

If the digital and analog clock becomes un-synchronized by accident, follow the steps below to restore the synchronization:

1. Press and hold  and the alarm switch (snooze) for 5 seconds. The display will light up and the hours will blink on the display.
2. Turn the jog wheel to set the hours so that they match the analog clock and press the jog wheel button to confirm. The minutes will blink on the display.
3. Turn the jog wheel to set the minutes so that they match the analog clock and press the jog wheel button to confirm.
4. The clock is now prepared for either automatic or manual time setting.

## Warranty and technical data

This product is guaranteed for a period of 24 months from the date of purchase. In the unlikely event of a fault occurring during this period, please contact the place of purchase. Proof of purchase is required for any service or support needed during the guarantee period.

This guarantee will not apply to a fault caused by an accident or a similar incident or damage, liquid ingress, negligence, abnormal usage, non-maintenance or any other circumstances on the user's part. Furthermore, this guarantee will not apply to any fault caused by a thunderstorm or any other voltage fluctuations. As a matter of precaution, we recommend disconnecting the device during a thunderstorm.

Batteries are consumables and are not included in any guarantee.

Use only AA Alkaline batteries of good quality.

### Specifications

Size: 114 mm x 127 mm x 94 mm

Weight: 567 g (including batteries)

Battery: 4 x 1.5 V AA Alkaline batteries

### Declaration of Conformity

Doro hereby declares that **Doro HearPlus 333cl** conforms to the essential requirements and other relevant regulations contained in the Directives 1999/5/EC (R&TTE) and 2002/95/EC (RoHS).

A copy of the Declaration of Conformity is available at  
[www.doro.com/dofc](http://www.doro.com/dofc)

English  
Swedish  
Norwegian  
Danish  
Finnish  
French  
German  
Spanish  
Italian  
Dutch  
Greek

Version 1.1

[www.doro.com](http://www.doro.com)

