Tactile Ring Signaler

Model: TRS-100

INSTRUCTION MANUAL

3RD EDITION

About Your TRS-100

Thank you for selecting your new TRS-100 Tactile Ring Signaler from Krown Manufacturing, Inc. It is designed to meet the highest quality standards and provide you with years of convenience and trouble-free services. The TRS-100 is designed especially for people with hearing and vision impairments.

- » It enables users to be aware when the telecommunication device is ringing.

This TRS-100 is user-friendly, dependable and pre-programmed.

Read these operating instructions thoroughly before using your TRS-100 and keep this guide for your future reference.

Warranty

Save your receipt as proof of purchase should you need warranty services. Also, mail in your Product Registration Card within ten days of purchase. While registration is not required for warranty service, the information you provide allows us to contact you with any new developments about your TRS-100's operation. It comes with a one-year warranty.

About this User Guide

This guide is designed to walk you through each feature step by step. Most only involve one or two steps and become automatic as you use them.

Getting Help

If you have any questions about your TRS-100, please call our Customer Relation Department at:

(800) 366-9950
(817) 738-2485
(817) 738-1970 Fax


PACKAGE CHECKLIST

Your TRS-100 package should contain the following items:

1. TRS-100 Transmitter: Small device with On/Off switch on the side with a telephone plug.
2. TRS-100 Pager with the belt clip: Device with a belt clip, On/Off switch and a 3.5mm jack for charger.
3. One 8.4V Rechargeable Battery: Similar to 9V battery size.
4. AC Adapter: Adapter plug with long power cord for connection to the 3.5mm jack of TRS-100 pager.
5. Operating Instructions

Description

The TRS-100 pager will vibrate when the transmitter is activated by an incoming call

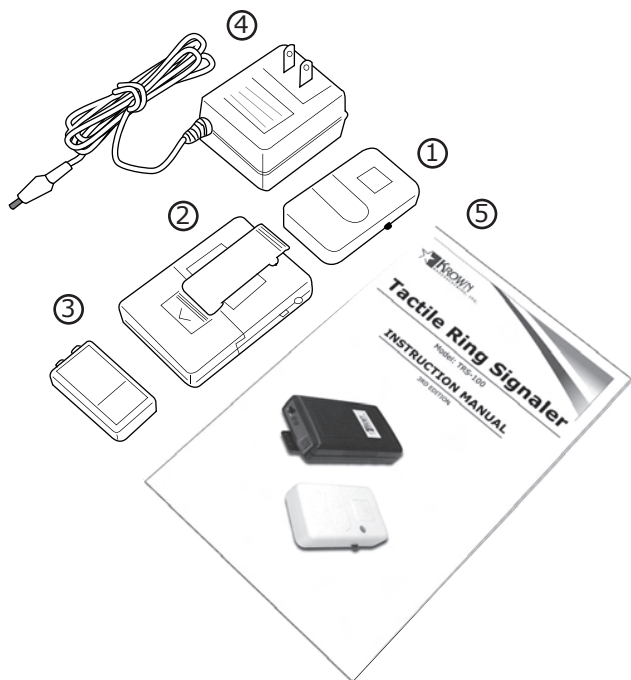IMPORTANT SAFETY INSTRUCTIONS

When using your Tactile Ring Signaler, basic safety precautions should always be followed to reduce the risk of fire, electric shock and persons including the following:

1. Read and understand all instructions.
2. Follow all warnings and instructions marked on the transmitter and pager.
3. Do not use the transmitter or pager near a bathtub, washbasin, kitchen sink or laundry tub, in a wet basement, near a swimming pool or anywhere else there is water.
4. Do not use the telecommunication device with this transmitter to report a gas leak in the vicinity of the leak.
5. Unplug the transmitter from the wall outlets, and the pager from the adapter cord before cleaning. Do not use liquid cleaners or aerosol cleaners on the telecommunication device. Use a damp cloth for cleaning.
6. Place the telecommunication device with the transmitter on a stable surface. Serious damage and/or injury may result if the telecommunication device falls.
7. The telecommunication device with the transmitter should never be placed near or over a radiator or heat register. This telecommunication device should not be placed in a built-in installation unless proper ventilation is provided.
8. Recharge this pager using the electrical voltage as stated on the base unit or the owner's manual. If you are not sure of the voltage in your home, consult your dealer or local power company.
9. Do not place anything on the power cord. Install the telecommunication device with the transmitter where no one will step or trip on the cord.
10. Do not overload wall outlets or extension cords as this can increase the risk of fire or electrical shock.
11. Never push any objects through the slots in the transmitter or pager, as touching dangerous voltage points or short out parts could result in fire or electrical shock. Never spill liquid of any kind on the transmitter and pager.
12. To reduce the risk of electrical shock, do not take this transmitter and pager apart. Opening or removing covers may expose you to dangerous voltages or other risks. Incorrect reassembly can cause electric shock when the appliance is subsequently used.

13. Unplug this transmitter from the wall outlets and refer servicing to the manufacturer under the following conditions:
 - When the power supply cord or plug is frayed or damaged.
 - If liquid has been spilled into the product.
 - If the telecommunication device with the transmitter or pager has been exposed to rain or water.
 - If the telecommunication device with the transmitter or pager does not operate normally by following the operating instructions. Adjust only those controls that are covered by the operating instructions. Improper adjustment may require extensive work by a qualified technician to restore the telecommunication device with transmitter and pager to normal operation.
 - If the transmitter or pager has been dropped or the case has been damaged.
 - If the transmitter or pager exhibits a distinct change in performance.
14. Never install this transmitter during a lightning storm.
15. Never install telecommunication device jacks in wet locations unless the jack is specifically designed for wet locations.
16. Never touch un-insulated telecommunication device wires or terminals unless the telecommunication device line has been disconnected at the network interface.
17. Use caution when installing or modifying telecommunication device lines.
18. Use only the power cord and batteries indicated in this manual. Do not dispose of batteries in a fire. They may explode. Check with local codes for possible special disposal instructions.

ADDITIONAL SAFETY NOTES FOR CANADIAN FOR CANADIAN USERS

The following items are included as part of the CS-03 Requirements. The standard connecting arrangement for the equipment is CA11A. This product meets the applicable Industry Canada technical specifications.

NOTICE: *The Canadian Department of Communications label identifies certified equipment. This certification means that the equipment meets certain telecommunications network protective operational and safety requirements. The Department does not guarantee that the equipment will operate to the user's satisfaction.*

Before installing this equipment, users should ensure that it is permissible to be connected to the facilities of the local telecommunications company. The equipment must also be installed using an acceptable method of connection. In some cases, the company's inside wiring associated with a single line individual service may be extended by means of a certified connector assembly (telecommunication device extension cord).

The customer should be aware that compliance with the above conditions might not prevent degradation of service in some situations. Repairs to certified equipment should be made by an authorized Canadian maintenance facility designated by the supplier. Any repairs or alterations made by the user to this equipment, or equipment malfunctions, may give the telecommunications company cause to request that the user disconnect the equipment.

Users should ensure for their own protection that the electrical ground connections of the power utility, telecommunication device lines and internal metallic water pipe system, if present, are connected together. This precaution may be particularly important in rural areas.

CAUTION: Users should not attempt to make such connections themselves, but should contact the appropriate electric inspection authority, or electrician, as appropriate.

The Ringer Equivalent Number is an indication of the maximum number of terminals allowed to be connected to a telecommunication device interface. The termination on an interface may consist of any combination of devices subject only to the requirement that the sum of the Ringer Equivalent Number of all the devices not exceed five.

BATTERY INSTALLATION

TRS-100 Pager

Install one 8.4V rechargeable battery into the battery compartment in the back of TRS- 100 pager. The batteries are used for operation. They power the TRS-100 pager for approximately 48 hours standby. Do not use Alkaline batteries, they will damage your TRS-100 pager.

Setting up the Tactile Phone Signaler

TRS-100 Transmitter

1. Plug the TRS-100 transmitter into the 2nd telephone jack of a telecommunication device. If your telecommunication device does not have an available jack for use, then use a telephone duplex adapter.
2. Turn on the transmitter by pushing the switch up on top.

TRS-100 Pager

1. Locate the On/Off switch on the top of the pager. The "ON" position is located the closest to the side of the top. Push the switch to "ON" position to turn it on.
2. You can wear it by using the belt clip on the pager. It vibrates when the transmitter is activated by an incoming call.

Charging the pager

When the battery is getting low, one LED on the top will flash and vibrate in pulse pattern. Then you should charge your pager with the TRS-100 adapter.

When the adapter is connected for charging, the low battery LED on top will turn on to indicate that the battery is in charging condition. Allow at least 4 hours of charging before using the pager.

WARRANTY PROCEDURE

Compu-TTY, Inc. warrants that the Tactile Ring Signaler sold by Krown Manufacturing, Inc. is free from defects in materials and workmanship under normal use and service for one year. This warranty is applicable only to the original purchaser of the Tactile Ring Signaler when accompanied by a sales receipt stating the date of purchase and the name of the company from which it was purchased. This warranty is in lieu of and excludes all other warranties, expressed or implied, including any implied warranty of **merchantability** or fitness and of any obligation on the part of Krown Manufacturing, Inc. If the Tactile Ring Signaler shall prove defective, then Krown Manufacturing, Inc. shall either replace or repair the Tactile Ring Signaler at no cost to the original purchaser within one year of the date of purchase. During the one-year period you may return the Tactile Ring Signaler to Compu-TTY, Inc. at the following address:

Krown Manufacturing, Inc.
3408 Indale Road, Fort Worth, TX 76116

Shipping charges are at the customer's expense. Please include a check or money order made out to Krown Manufacturing in the amount of \$10.00 to cover shipping and handling. This warranty does not extend to any Tactile Ring Signaler sold by Krown Manufacturing, Inc., which has been subject to misuse, neglect, accident, improper application or installation, or any use in violation of the instruction furnished by Krown Manufacturing, Inc. This warranty does not extend to or apply to any Tactile Ring Signaler, which has been repaired or altered by any persons who have not been expressly approved by Krown Manufacturing, Inc.

This warranty does not apply to the Tactile Ring Signaler if it has been damaged, modified, abused or altered after the date of purchase or it fails to operate due to improper maintenance or inadequate DC power.

In no event shall Krown Manufacturing, Inc. be liable for loss of this product or for any indirect, special, incidental, or consequential damages, personal injury, property damage, or cost or expenses incurred by the consumer or any other user of this product. This warranty is only applicable to a product purchased through Krown Manufacturing, Compu-TTY or a Krown dealer. Other than stated herein, there is no other warranty whether by the sample or model, by description or quantity, whether expressed or implied, and there are no implied warranties of **fitness for a particular purpose or of merchantability.**

TRS-100 TELEPHONE RING SIGNALER

Specifications

Operates on analog telephone lines

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