

WARRANTY

The Silent Call AlarmDock is warranted for (see individual items for length of warranty) from the day of purchase to be free of defects in material and workmanship. This warranty is limited to free repair or replacement when the product is shipped prepaid to:

Silent Call® Communications Corporation
5095 Williams Lake Road
Waterford, Michigan 48329

This warranty is void when damage is caused by misuse, neglect, fire, excessive heat, water, or by improper installation or operation. Silent Call will not be responsible for the cost of parts or labor and/or service charges which are performed by others.

To obtain warranty service, call toll free 1-800-572-5227 or email us at technicalsupport@silentcall.com

After this warranty expires, Silent Call will repair or replace any Silent Call product for a fixed cost.

www.silentcall.com
Silent Call® Communications Corporation
5095 Williams Lake Road
Waterford, Michigan 48329

SilentCall
Communications

SilentCall
ALARMDOCK

Smartphone Docking Station with Speaker and Bed Shaker

For use with any Smartphone or Tablet



User's Manual

TABLE OF CONTENTS

Introduction.....	4
Important Safety Instructions.....	4
Danger	4
Warnings	4
Precautions.....	4
Specifications USB Power Cube.....	4
Smartphone Application Requirements.....	4
Manual Reviews	5
AlarmDock Specifications	5
AlarmDock Cradle, Model # AD100SM	6
Application and Setup.....	7
Bed Shaker, Model # VIB-PJ.....	7
Wireless Bed Shaker Module, # AD200-SM	7
USB Charger/Power Cube, Model # 890010	8
Shaker Module Setup and Pairing.....	8
Pairing Instructions for the Bed Shaker	9
FCC Statement.....	9
AlarmDock Application Setup Icons.....	9
Troubleshooting Tips.....	10
Warranty.....	back cover

INTRODUCTION

Thank you for choosing the AlarmDock™ Smartphone speaker. This instruction manual contains useful information on appropriate use and maintenance of this product. In order to benefit from its features, read this manual before first use and save it for further reference.

IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING: Read all instructions before using this product.

DANGER:

- Do not let water or any other liquid or flammable detergent enter the power jack to avoid electric shock and/ or fire hazard. Do not clean the power cable with water or any other liquid or a (flammable) detergent to avoid electric shock and/or a fire hazard.
- To prevent fire or shock hazard do not expose this product to water, or moisture. Before connecting to the main power make sure that the requirements specified on the power cube unit corresponds to the available power. i.e. AC100-240V~, 50/60 Hz.

WARNINGS

- This product is not intended for use by children or persons with reduced physical, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the unit by a person responsible for their safety.
- Modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

PRECAUTIONS

Children should be supervised to ensure that they DO NOT play with the unit especially the parts.

SPECIFICATIONS USB Power Cube

Switching type AC/DC Power Adapter, Rated Input: 100-240 Vac, 50/60 Hz, 0.8A

SMARTPHONE APPLICATION REQUIREMENTS

Androids: 4.0 or higher

iPhone: iOS 8 or later, compatible with iPhone, iPad, and iPod touch.

App size 2.4 MB.

THIS MANUAL REVIEWS PROPER CONNECTION AND USE OF:

- AlarmDock cradle with built-in speaker, Model # AD100SM
- Bed Shaker, Model # VIB-PJ
- Shaker Module, Model # AD200-SM
- USB Charger/ Power Cube, Model # 890010
- AlarmDock smartphone application (iOS and Android)

ALARMDOCK SPECIFICATIONS

Speakers: 2 x 40mm

Amplifier power: 2 x 4w

Frequency response: 150Hz - 20kHz

Signal-to-noise ratio: up to 100dB

Distortion: ≤5%

Battery type: polymer Li-ion

Wireless transmitter frequency range: 2.4–2.48GHz

Battery charging voltage: 5V

Battery charge time: 3-4 hours


Battery capacity: 1000 mAh

Wireless transmission distance: 33 feet

Dimensions: 6.2 x 4.2 x 5.6 inches

1 ALARMDOCK CRADLE, MODEL # AD100SM, FEATURES:

- 2-in-1, wireless speaker and phone stand
- Extra-loud alarm, up to 100 decibels
- Unique, patented design
- Touch-sensitive controls
- 2 broad-frequency response speakers
- Rich, 360°, crystal-clear sound
- Compatible with Android/iOS/Windows/MAC OS
- Built-in, rechargeable Li-ion battery provides up to 10 hours play time
- Wireless and audio (port) connections – aux. cord not included
- Memory pairing capability. Easy to operate
- Two-way USB port, plays and charges at the same time
- 90 day warranty

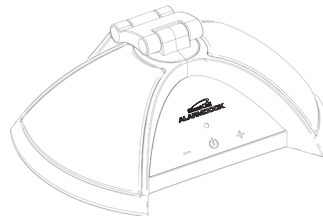


APPLICATION AND SETUP

You'll need to download the AlarmDock™ application. For Android's go to the Google play store. For iPhone go to the iTunes App Store. Type AlarmDock™ into the application search field, then look for AlarmDock by Silent Call® Communications. Once you've downloaded the application, you'll need to open it to setup your alarm settings and to pair the wireless bed shaker. To use the AlarmDock™ application one device at a minimum must be paired for it to work. See application requirements under smartphone application requirements above.



2 BED SHAKER, MODEL # VIB-PJ, 5 YEAR WARRANTY

Best placement for the bed shaker is under the mattress, between the mattress and box spring. Place in a spot that would be closest to your upper back. Once in place, plug the bed shaker into the end of the Shaker Module input jack. Plug the Shaker Module into the USB charger cube, then plug the USB charger cube into the 120 volt wall outlet. Very important "DO NOT USE ANY OTHER MANUFACTURERS BED SHAKER WITH THE Shaker Module, IT COULD DAMAGE THE SHAKER MODULE AND VOID THE WARRANTY".

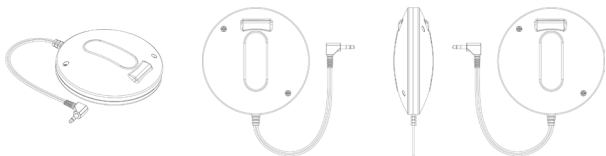


3 BED SHAKER MODULE, # AD200-SM, 90 DAY WARRANTY

Plug the wireless bed shaker module into the USB charger cube. Plug the USB charger cube into a 120 volt wall outlet. The USB charger cube will keep the Shaker Module powered and ready to activate the bed shaker. When the Shaker Module is in range, approximately 30 feet from your smartphone it will automatically pair itself to the smartphone. A blue light on the Shaker Module will light up and flash on & off when ready to be paired. When paired the blue light will stay steady, no flashing.

4 USB CHARGER/POWER CUBE, MODEL # 890010, 1 YEAR WARRANTY

The USB charger Cube is designed to power and charge up to four devices at one time. Charge devices like the AlarmDock™ wireless speaker stand, cell phones, & tablets. It also powers the wireless bed shaker module.

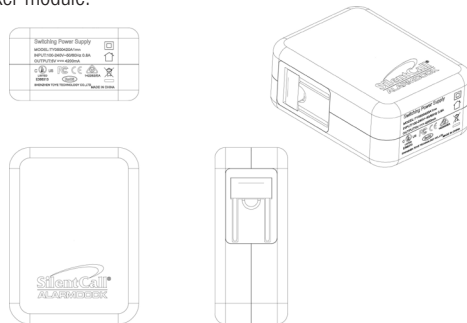

SHAKER MODULE SETUP AND PAIRING

Plug the bed shaker jack into the jack input at the end of the Shaker Module. Next plug the Shaker Module into the USB Charger Cube and plug the USB Charger Cube into any 120 volt outlet. Make sure the jack on the bed shaker is pushed all the way into the Shaker Module. Place the bed shaker between the mattress and box spring of your bed. For best result, place the bed shaker under the area where your upper back would be positioned when in bed.



Note: Different diagram needed

PAIRING INSTRUCTIONS FOR THE BED SHAKER

Bed Shaker Pairing: Once your smartphone is in range of the Bed Shaker approximately 30' or less the Bed Shaker will automatically pair with your smartphone. A blue flashing indicator light will flash on the Shaker Module stick and will change to steady lit no flashing when paired. If the blue LED is still flashing, you'll need to turn on the Bluetooth setting in your phones settings.

FEDERAL COMMUNICATIONS COMMISSION (FCC) STATEMENT

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Warning: Changes or modifications made to this device not expressly approved by Silent Call Communications may void the FCC authorization to operate this device.

Note: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

ALARMDOCK™ APPLICATION SETUP ICONS (LOCATED AT THE BOTTOM OF THE APP)

There are five icons located at the bottom of the application, below we explain each individually.

Connect icon (pairing the wireless bed shaker/Vibrator Shaker Module)

Connect icon (pairing with AlarmDock wireless cradle)

Alarm Icon – Tap on the Alarm icon, then tap the + at the top right corner to setup an alarm. Follow the app instructions. Here you can set multiple alarms, repeat the alarms with days of the week selections. You can name the alarm with labels, select the sound type, volume control, tone control, and select an alarm fade option from 10 seconds to 5 minutes or none.

Timer icon – Tap the Timer icon to set a timer.

Clock Icon with Swipe to Dim Feature – Tap on the Clock icon swipe on the display screen in either direction to select from eight different display colors. Once the color has been selected you can dim the display by swiping left to right or top to bottom.

Settings (located at the bottom right-hand side) – To leave the clock display, tap anywhere on the screen. Tap settings to see all the choices available. Adjust the volume, vibration strength. Select how you want to be alerted, Sound, vibrator or your phones flashing light. Adjust the snooze duration from 5 minutes to 50 minutes, and select the preferred time format 12/24 hour.

TROUBLESHOOTING TIPS

Please read this User Guide carefully before using your wireless speaker. Check this list for a possible troubleshooting solution before calling for service.

Problem	Solution
No power	<ul style="list-style-type: none">• Make sure that the USB charging line is correctly connected.• The built-in lithium battery may be drained. Charge the battery by connecting the USB charger.
No sound output via wireless	<ul style="list-style-type: none">• Make sure your wireless device is properly connected to the speaker.• Make sure that the volume level of your AlarmDock device is not set to the minimum.• Make sure that the volume level of your wireless device connected to the speaker is not set to the minimum.• Make sure that your wireless device is not muted.• Make sure that your stereo audio cable is not connected to the AUX IN jack of the speaker.
No sound output in AUX IN mode	<ul style="list-style-type: none">• Make sure that the AUX cable is undamaged and firmly connected to the AlarmDock device.• Make sure that the volume level of the speaker is not set to the minimum.• Make sure that the volume level of the connected external device is not set to the minimum.
Low battery warning	<ul style="list-style-type: none">• When the battery is low, the indicator light will turn red and an alert will sound. Use the USB cable for charging.• The internal battery is drained. Charge the battery or connect the USB charger and turn on the system again.

Problem	Solution
Wireless connection cannot be established	<ul style="list-style-type: none">• Make sure that the audio cable has been removed from the AUX IN jack of your wireless speaker.• If the connection fails when making a wireless connection between the speaker and your wireless device for the first time, turn off your AlarmDock unit, then turn it on again to clear the device name, then try the connection again.
Wireless Shaker Module won't stop flashing	<ul style="list-style-type: none">• Turn on your Bluetooth setting in your phones settings. Also make sure your phone is in range, 30' or less.
Sound, vibration or flashing light not working	<ul style="list-style-type: none">• Go to the app settings icon and turn on the alert.
Wireless vibe/bed shaker too weak or not working	<ul style="list-style-type: none">• Go to settings and slide the vibration slider to the right.• Check that the Bed Shaker plugged in all the way to the Shaker Module.• Make sure that the Shaker Module is plugged in all the way on the power/charger cube.