


Installation and Operating Instructions

Doorbell Transmitter (US version)

Model DB1-SS

Installation

1. Locate your doorbell chime unit and remove the cover.
2. Inside the chime unit, locate the terminal block with three terminals marked FRONT, REAR and TRANS.
3. Loosen the two screws but do not disconnect the attached wires.
4. Attach the two wires from the DB1-SS as follows: Red to FRONT, and yellow to REAR. The black to TRANS terminal is not used.
5. Tighten the two screws and tuck and excess wires out of sight. Make sure all wires are clear of the chime mechanism and reattached the chime unit cover.
6. Using the supplied Velcro strip, attach the Doorbell transmitter to the wall next to the chime unit or to the chime unit cover. You can also hang the transmitter on a nail or screw using the opening in the back of the case.

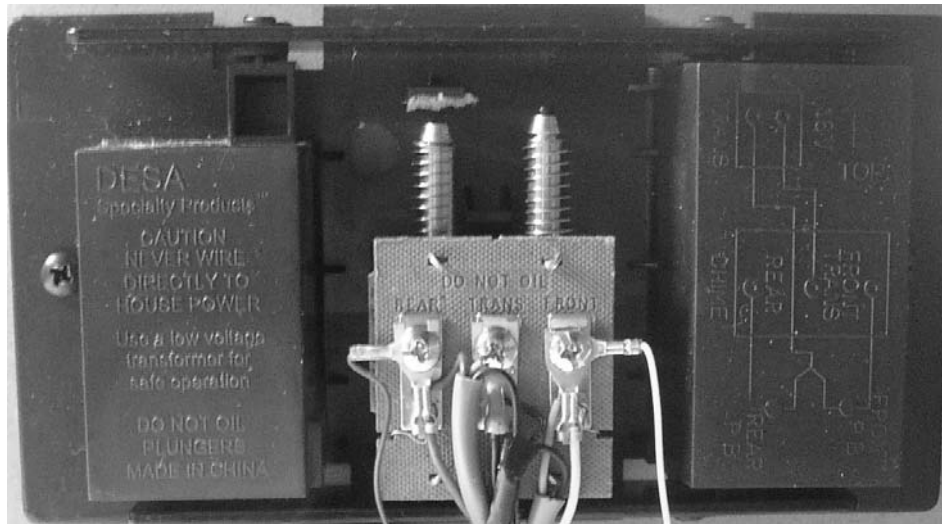

Operation

1. Whenever the front (or rear, if used) doorbell button is pressed, the chimes will sound as usual AND the Red LED on the doorbell transmitter will light up. The transmitter then sends a signal to any Signature Series Receiver activating the receiver.
2. Transmission range is determined by which Signature Series receiver you are using.
3. Both the front and rear doorbells will activate the doorbell transmitter.
4. This unit is powered by the doorbell transformer and does not require a battery. The doorbell transformer should be in the 12 -24V range.



5095 Williams Lake Road, Waterford Michigan 48329

(800) 572-5227 v/tty (248) 673-7360 fax

Website: www.silentcall.com Email: silentcall@silentcall.com

Address Switch Settings

The Silent Call system is digitally encoded. All Silent Call receivers and transmitters are tested and leave the factory programmed to a factory default address. ***You do not need to change the address unless someone in your area has Silent Call products and they are interfering with your equipment.***

1. Make sure that all Silent Call transmitters in the area are turned off.
2. Disconnect the Doorbell transmitter from the chime box. Note that you MUST disconnect the transmitter from the chime box first or the switch setting will not take effect.
3. Located on the back of the transmitter case is a removable access panel. Remove the access panel.
4. Locate the address switch on the transmitter circuit board that has 5 small dip switches. Set the switches to any combination that you want. For Example: 1, 2 ON 3, 4, 5 OFF. This gives your transmitter an "address". Note: Do NOT set the switches to the all "ON" or all "OFF" position.
5. Reinstall the access panel on the back of the transmitter and reattached the transmitter to the doorbell chime box.
6. Refer to your specific Signature Series Receiver instruction manual for programming your receiver to your newly changed transmitter address.

Technical Support

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at support@silentcall.com

Limited Warranty

Your transmitter is warranted to be free of defects in material and workmanship for five years from the date of initial purchase. During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. This warranty is void if the defect is caused by customer abuse or neglect.



5095 Williams Lake Road, Waterford Michigan 48329

(800) 572-5227 v/tty (248) 673-7360 fax

Website: www.silentcall.com Email: silentcall@silentcall.com