

Installation and Operating Instructions

Model DB2-MC

Doorbell Transmitter with Remote Button 418MHz

Installation:

1. Decide where to mount the transmitter on an inside wall near the location for the button.
2. Drill a hole in the wall behind where the transmitter will mount.
3. Pass the wires from the transmitter through the hole and connect them to the terminals in the button.
4. Install the button on the outside wall covering the hole.
5. Mount the transmitter to the wall over the hole using the supplied Velcro strip or you can also hang the transmitter on a nail or screw using the opening in the back of the case.

Operation:

1. When the remote button is pressed, the Red LED on the face of the transmitter will light up. The transmitter will then send a signal to any Silent Call Medallion Series Receiver activating the receiver.
2. Transmission range with a Silent Call Medallion Series receiver is about 500 feet.
3. This unit is powered by two AA alkaline batteries (included) which should last a year or more, depending on use.
4. There is a Yellow LED (low battery indicator light) on the face of the transmitter to let you know the battery is low and needs to be changed.

Transmitter Settings

The Silent Call Medallion system is digitally encoded. All Silent Call Medallion Series receivers are capable of receiving one million different codes and the transmitters are preset at the factory to send one of the million different codes. Please refer to your receiver manual for instruction on teaching the transmitter code to the receiver.

Replacement Batteries:

Use two AA 1.5V Alkaline Batteries. AA Batteries may be purchased from local retail stores. Transmitter is shipped with 2 Rayovac Alkaline Batteries Type LR6. Batteries may be purchased Online at www.silentcall.com.

Technical Support

For technical support on this or any other Silent Call product, please feel free to contact us. You can reach us by phone at 800-572-5227 (voice or TTY) or by Email at support@silentcall.com

Limited Warranty

Your transmitter is warranted to be free of defects in material and workmanship for five years from the date of initial purchase. During that time, the unit will be repaired or replaced free of charge when shipped prepaid to Silent Call Communications. This warranty is void if the defect is caused by customer abuse or neglect.



Return for Serving To:

Silent Call Communications

5095 Williams Lake Road, Waterford Michigan 48329

(800) 572-5227 v/tty (248) 673-7360 fax

Website: www.silentcall.com Email: silentcall@silentcall.com