

99.00 mm

Model: SBH400SS (RHK205)

NO:093-HK205-006 (12HR Only)

風琴式 二邊印

English



Sonic Alert
America's Leader in Visual Alerting Systems

Sonic Boom Alarm Clock & Bed Shaker
Model SBH400SS



OPERATION INSTRUCTIONS



IMPORTANT- Please read these instructions carefully before use and retain for future reference. See operating instructions on back.
- Use only the supplied adaptor to connect unit

- Warning:** To prevent fire or shock hazard do not expose this appliance to rain or moisture. Before connecting to mains make sure that the requirements specified on the apparatus corresponds to the available power supply i.e. AC120V~ 60 Hz
- The ventilation should not be impeded by covering the ventilation openings with items, such as newspaper, table cloths, curtains, etc;
 - No naked flame sources, such as lighted candles, should be placed on the apparatus.
 - It shall not be exposed to dripping or splashing and that no objects filled with liquids, such as vases, shall be placed on the apparatus.
 - Do not place the product in closed bookcases or racks without proper ventilation.
 - Where the mains plug is used as the disconnect device, the disconnect device shall remain readily operable..

GENERAL DATA
Power: Consumption: AC120V~ 60 Hz, 8W (for USA)
Adaptor Output: AC9V~ 900 mA
UL listed Adaptor (Underwriters Laboratories) for USA.

Made in China
Sonic Alert 1050 East Maple Rd, Troy, MI 48083
www.SonicAlert.com 1-800-566-3210 V/TTY

INSTRUCTIONS LOUD -VIBRATING ALARM CLOCK WITH BED SHAKER MODEL: SBH400SS

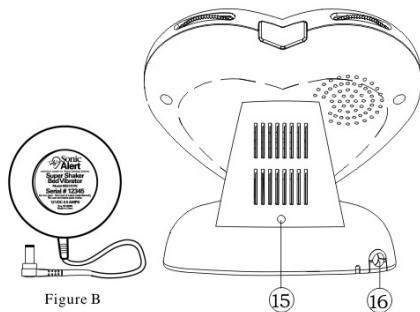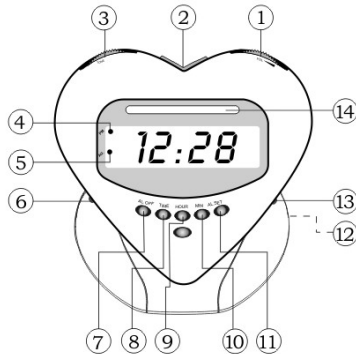

Features :

- | | |
|---------------------|--------------------------------|
| 1) Volume Control | 10) Minute Button |
| 2) Snooze Button | 11) Alarm Set Button |
| 3) Tone Control | 12) Backup Battery Compartment |
| 4) PM Indicator | 13) Function Switch |
| 5) Alarm Indicator | 14) Flashing Display |
| 6) Dimmer Switch | 15) AC Power Input |
| 7) Alarm Off Button | 16) Vibrator Input |
| 8) Time Set Button | |
| 9) Hour Button | |

Setting the correct time-

- 1) Press and hold the "TIME" set (8) button.
- 2) Depress hour button (9) as necessary to advance the clock display until the hour is correct.
- 3) Depress minute button (10) as necessary to advance the clock display until the minute is correct.
- 4) Once the time is set, release the "TIME" set (8) button.
- 5) Lighted dot (4) in upper left clock display indicates time is PM, a dot does not appear for AM.

Setting the alarm time-

- 1) Simply follow the instructions mentioned under " Setting the correct time" except that you will have to Press and hold the "ALARM" set (11) button.
- 2) Make sure the alarm time is correctly set for AM or PM lighted dot (4) indicates PM. Be sure to release the HOUR and MIN set buttons before releasing the ALARM button.
- 3) Once the time is set, release the "ALARM" set (11) button.
- 4) To enable the alarm function, slide the Function Switch (13) to the desired position. This will turn on the alarm feature. Otherwise, alarm will not go off for the alarm time set.
- 5) You will see a dot (5) on the left side of the LED display to show the alarm has been enabled.

Alarm-

Slide the "FUNCTION SWITCH" (13) up to the position that you desire to wake to, you can choose "VIB" for bed shaker, "BUZZ" for sound, and "VIB/BUZZ" for both. NOTE: whichever function you choose to wake to, will be the same function for both alarms. When the alarm is activated, the Flashing Display (14) will flash until the alarm is stopped.

Snooze Operation-

SNOOZE button (2) - press after alarm has sounded. The alarm will be turned off for 9minutes for extra sleep. The alarm will come back again after 9minutes.

Shutting Off Alarm-

Alarm can be shut off temporary for the day or permanently shut off.

- 1) When the alarm goes off, simply press the AL off button (7) the alarm will shut itself off for the day and come back on the next day at the same time if alarm setting has not changed
- 2) To shut the alarm off permanently, slide Function Switch (13) to off position. Alarm will not come on until Function Switch (13) is set to "BUZZ" or "VIB" or "VIB/BUZZ" position.

Volume Control-

To adjust the volume rotate turn the wheel located on top right side (1) of the clock.

Tone Control-

To adjust the "TONE CONTROL" (3) turn the wheel located on top left side (3) of the clock to the desired tone sound.

Bed Shaker / Vibrator- Figure B

- 1) Attach the bed shaker jack to the back of clock at VIBRATOR input (16).
- 2) To turn on vibrator slide FUNCTION Switch (13) to VIB or VIB/BUZZ position.
- 3) Place vibrator under mattress or pillow.

Battery Backup-

To use battery backup, you can install a 9-volt alkaline battery in the bottom of the clock located under the battery cover (12). Battery not included. Note the LED display will not show during power failure but the clock will still be running and will maintain proper time and alarm settings until the power is restored. NOTE: The battery backup only works for a short periods only, about two hours.

AC-9V Power Input -

Attach the AC power supply input to the back of clock at AC-9V IN (15) next to the VIBRATOR input, smaller input of the two.

NOTE: IMPORTANT: do not confuse "AC POWER INPUT"(15) with the "VIBRATOR INPUT"(16).



The Lightning flash and arrowhead within the triangle is a warning sign alerting you of dangerous voltage inside the product.

CAUTION:
TO REDUCE THE RISK OF ELECTRICAL SHOCK, DO NOT REMOVE THE COVER (OR BACK) FROM THE PRODUCT. NO USER SERVICEABLE PARTS INSIDE. REFER SERVICING TO YOUR SONIC ALERT DEALER.



The exclamation mark within the triangle is a warning sign alerting you of important instructions accompanying the product.

WARNING: TO PREVENT FIRE OR SHOCK HAZARD, DO NOT EXPOSE THIS APPLIANCE TO RAIN OR MOISTURE.

SONIC ALERT'S 1 YEAR LIMITED WARRANTY

The Sonic Boom Alarm Clock Model SBH400ss is warranted against manufacturing defects in materials and workmanship for (1) year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the SBH400ss without charge for parts and labor. Simply send the SBH400ss (postpaid) and a copy of your sales slip as proof of purchase to:

Sonic Alert Inc., 1050 E. Maple Road, Troy MI 48083

This device complies with part 15 of the Federal Communications Commission Rules. Operation is subject to the condition that this device does not cause harmful interference.

Note: The manufacturer is not responsible for any radio or TV interference cause by unauthorized modifications to this equipment. Such modification could void the user's authority to operate the equipment.

- Attention should be drawn to the environmental aspects of battery disposal.

- this indicating separate collection for electrical and electronic equipment.



Sonic Alert
America's Leader in Visual Alerting Systems

Made in China

Sonic Alert 1050 East Maple Rd. Troy, MI 48083
www.SonicAlert.com 1-800-566-3210 V/TTY



093-HK205-006 (12 HR Only)