




TV·EARS®

Dr. Recommended TV Listening Devices

Removing and Installing TV Ears Tips

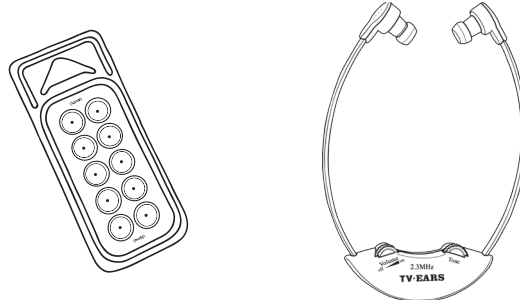

You'll need your ear tips and your headset


Note: Ear tips are recommended to be changed every 30-60 days.

Removing TV Ears Tips

To remove the ear tips from your headset, use the triangular tool located at the top of the plastic ear tip tray.

Step 1 1

Place the ear tip inside the triangular opening so that the plastic tool is wedged between the foam and the snap tip adapter as shown below.

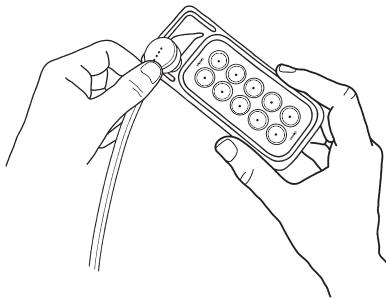

Step 1 2



Once the ear tip is properly placed in the triangular opening, grip the round part of the headset and gently pull the headset away from the ear tip. The ear tip will come off.

The picture above shows what the headset should look like after the ear tip is removed.

Note: If a small cardboard ring remains on the snap tip adapter, remove it prior to installing your new ear tips

Removing TV Ears Tips

To remove the ear tips from your headset, use the triangular tool located at the top of the plastic ear tip tray.

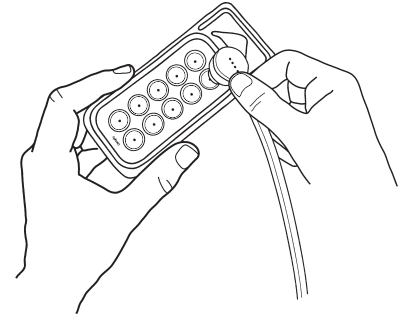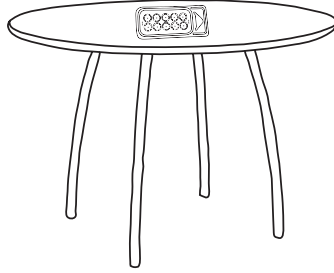

Step 1 1

Place the snap adapter on the headset. (may already be in place)

Step 1 2

Place the "TV Ears Tips" tray on firm flat surface or on table.

Step 1 3

Now that the plastic adapter is on the headset, push the tip of the headset into the center of a new tip until it clicks onto the headset.